INADA CHAIR

# YUMEROBO

## Rotary Rocker Technology

Thank you for purchasing the INADA CHAIR yUMEROBO Rotary Rocker Technology. You can enjoy various types of massage for relieving or soothing fatigue. We hope the INADA CHAIR yUMEROBO Rotary Rocker Technology will be a valued partner in your daily health preservation for many years to come.

感謝您購買 INADA CHAIR yUMEROBO Rotary Rocker Technology。本機能消除疲勞及舒展筋絡，帶給您多重按摩享受。希望 INADA CHAIR yUMEROBO Rotary Rocker Technology 能成為您永久的健康伴侶。

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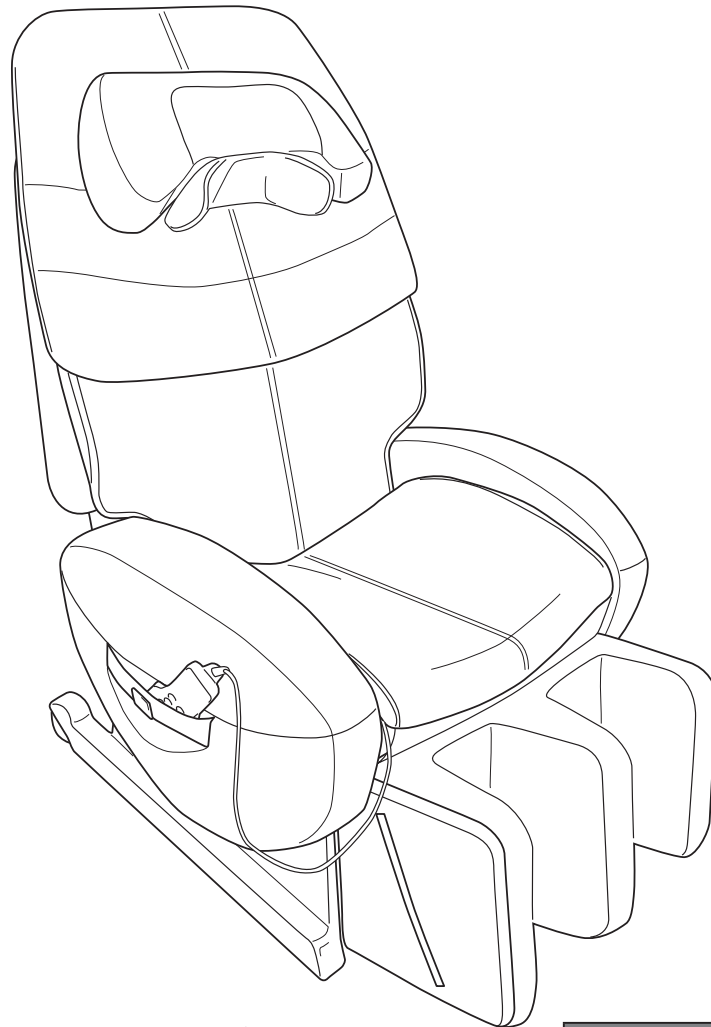
Operating Manual

Warranty card provided separately

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**NOTE**

---

INADA CHAIR

# YUMEROBO

## Rotary Rocker Technology

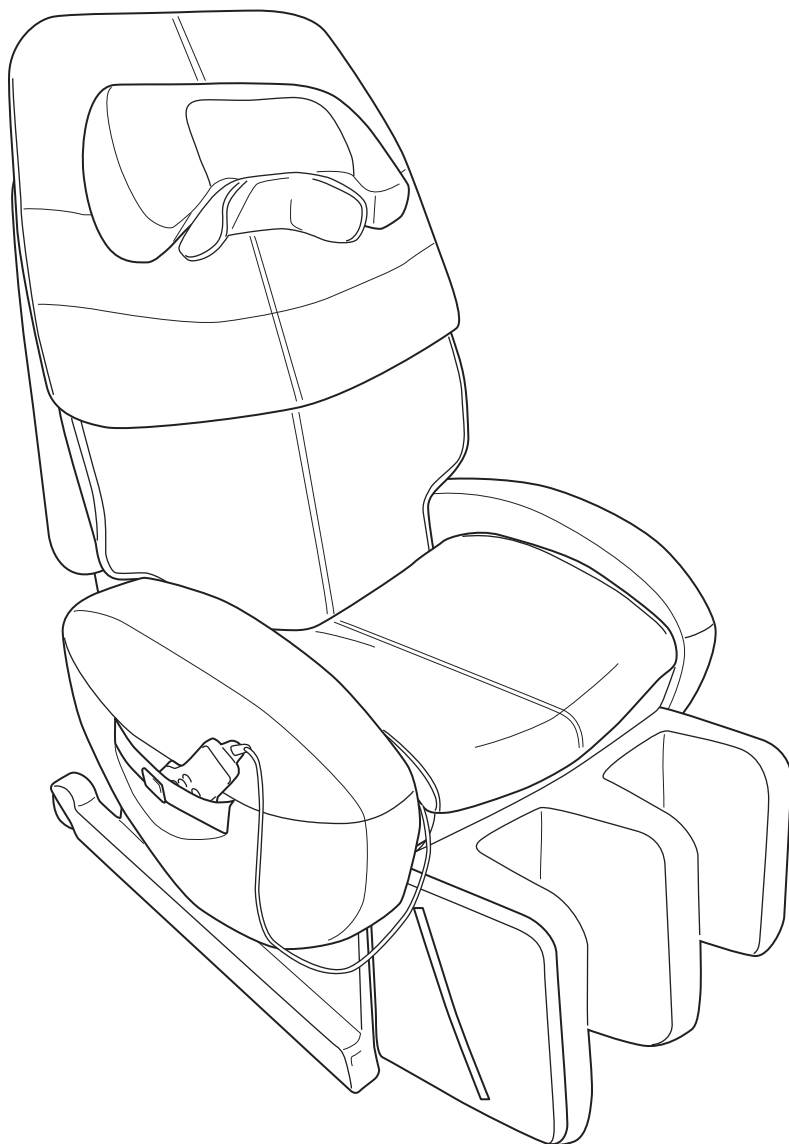
Thank you for purchasing the INADA CHAIR yUMEROBO Rotary Rocker Technology. You can enjoy various types of massage for relieving or soothing fatigue.

We hope the INADA CHAIR yUMEROBO Rotary Rocker Technology will be a valued partner in your daily health preservation for many years to come.

ENGLISH

HCP-R100D

### CONTENTS

■ SAFETY PRECAUTIONS .....	1
■ MAIN FEATURES .....	4
■ COMPONENTS & DESCRIPTION..	5
■ BEFORE USE .....	7
■ POSITION ADJUSTMENT .....	10
■ AFTER USE .....	13
■ PROGRAMS & SELECTION .....	14
■ STARTING A MASSAGE .....	15
■ MASSAGE ADJUSTMENTS .....	17
■ TROUBLESHOOTING .....	19
■ REGULAR MAINTENANCE FOR SAFETY .....	20
■ FAQ .....	21
■ CARE AND STORAGE.....	22
■ CUSTOMER SERVICE AND WARRANTY .....	22
■ SPECIFICATIONS .....	23

Operating Manual

Warranty card provided separately

- Please read this Operating Manual before use.
- Keep this Operating Manual nearby so that you can refer to it anytime.
- This product is designed for home use.

# SAFETY PRECAUTIONS

Be sure that you have read these SAFETY PRECAUTIONS thoroughly before using your INADA CHAIR yUMEROBO Rotary Rocker Technology for the first time. The Operating Manual contains important information about how to use the chair safely and correctly, avoiding possible damage to property or injury to persons using the chair and others. First of all, you should understand well the meaning of the indications and symbols described below, which are used throughout the Operating Manual to highlight important safety instructions. Please adhere to all precautions. Descriptions of each chair component can be found on pages 5-6.

## Types of Indications

 <b>Warning</b>	This action could result in serious injury or death.
 <b>Caution</b>	This action could result in personal injury or damage to property.

## Types of Symbols and Description

 <b>Prohibited</b> (The symbol on the left means "Do not disassemble.")
 <b>Required Action</b> (The symbol on the left means "Unplug the chair.")

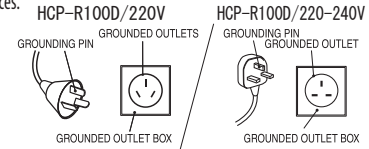
**Remember to keep this Operating Manual in an easily accessible place.**

## Safety Precautions: Installation

 <b>Caution</b>	
 <b>Prohibited</b>	<ul style="list-style-type: none"> <li>Do not install the chair in an area of high humidity, such as a bathroom or sauna. → Could result in electric shock or damage to the chair.</li> </ul>
 <b>Required Action</b>	<ul style="list-style-type: none"> <li>Install on a flat, level surface only. → Chair could tip over accidentally if placed on an uneven surface.</li> </ul>

## Safety Precautions: Before Use

 <b>Warning</b>	
 <b>Electrical grounding</b>	<ul style="list-style-type: none"> <li><b>Space Requirements</b> This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electric current in order to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances. Check with a qualified electrician or serviceman if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product if it will not fit the outlet; have a proper outlet installed by a qualified electrician. This product uses on a normal Alternating Current (AC) circuit and has a grounding plug that looks like the plug illustrated in the figure to the right. Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product.</li> </ul>
 <b>Required Action</b>	<ul style="list-style-type: none"> <li>Always unplug this appliance from the electrical outlet immediately after using and before cleaning.</li> </ul>

 <b>Caution</b>	
 <b>Prohibited</b>	<ul style="list-style-type: none"> <li>Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.</li> <li>Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.</li> <li>Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair.</li> <li>Do not carry this appliance by supply cord or use cord as a handle.</li> <li>Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.</li> <li>Never drop or insert any object into any opening.</li> <li>Do not use outdoors.</li> <li>Do not operate where aerosol (spray) products are being used or where oxygen is being administered.</li> <li>Children should be supervised to ensure that they do not play with the appliance.</li> </ul>
 <b>Required Action</b>	<ul style="list-style-type: none"> <li>An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.</li> <li>Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.</li> <li>Keep the cord away from heated surfaces.</li> <li>To disconnect, turn all controls to the off position, then remove plug from outlet.</li> <li>Keep children away from extended foot support (or other similar parts).</li> <li>Connect this appliance to a properly grounded outlet only. See Grounding Instructions.</li> <li>This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.</li> </ul>
 <b>Prohibited</b>	<ul style="list-style-type: none"> <li>The following persons must not use this massage chair: <ul style="list-style-type: none"> <li>Persons who have been instructed by a doctor to avoid massage therapy. Ex. Thrombosis, severe aneurism, acute varicosity, or dermatitis and/or skin infection including inflammation of the hypodermis.</li> <li>Persons with suspected acute neck sprain (whiplash injury)</li> </ul> </li> </ul>
 <b>Required Action</b>	<ul style="list-style-type: none"> <li>Persons with any of the following conditions should seek medical advice before using the chair: <ul style="list-style-type: none"> <li>Serious heart conditions</li> <li>Sensory impairment caused by serious peripheral circulatory disorders due to diabetes and other diseases</li> <li>Osteoporosis, spinal fractures, acute pain and/or symptoms thereof</li> <li>Persons being treated for personal injuries</li> <li>High fever (38°C (100°F) or higher) Ex. Strong acute inflammation symptoms (fatigue, chills, blood pressure fluctuation) or debilitated</li> <li>Persons in the early stages of pregnancy, or immediately after giving birth</li> <li>Persons who have electronic medical device(s) easily susceptible to electromagnetic interference Ex. pacemaker</li> <li>Abnormal or curved spine</li> <li>Persons planning to use the chair for treatment or for rehabilitation purposes</li> <li>Persons with malignant tumor(s)</li> <li>Persons under bed rest orders</li> <li>Persons with acute gastro-intestinal complaints or ailments such as gastritis, enteritis, or hepatitis</li> <li>Persons under a doctor's care, or those experiencing unusual physical discomfort</li> <li>Joint dysfunction due to rheumatism, gout, or other diseases</li> <li>Persons with symptoms such as qualm, dizziness, or tinnitus</li> <li>Inflammation due to sprain or torn muscle</li> <li>Persons with thecitis or suspected thecitis</li> </ul> </li> <li>Elderly people and persons with weak bones should seek medical advice before using the chair, even if they do not have a specific disease.</li> <li>If your condition does not improve after use, consult a doctor or specialist.</li> </ul>

[SAFETY PRECAUTIONS]

## Safety Precautions: Before Use

 <b>Caution</b>	
 Prohibited	<ul style="list-style-type: none"> <li>● <b>Do not use any other power supplies except alternating voltage 220-240V for HCP-R100D.</b> → May result in fire or electric shock.</li> <li>● <b>Do not use with any type of transformer device.</b> → May result in damage to the chair or electric shock.</li> <li>● <b>Do not use the chair, if the power cord or plug is damaged, or if the wall outlet connection is loose.</b> → May result in electric shock, short circuit, or combustion.</li> <li>● <b>Before each use, check for holes or tears in the chair covering. Also, check the other parts for tearing. Stop using the chair immediately if you find even the smallest tear. Unplug the chair and contact the vendor for assistance.</b> → Using the chair with holes or tears in the covering could result in injury or electric shock.</li> </ul>
 Required Action	<ul style="list-style-type: none"> <li>● <b>Before plugging the power cord into the wall outlet, make sure that the main POWER switch is set to OFF.</b> → May result in damage to the chair or electric shock.</li> <li>● <b>Make sure that the plugs on the power cord are inserted fully into the cord connector and the wall outlet.</b> → May result in electric shock, short circuit, or combustion.</li> <li>● <b>Before using the chair for the first time, turn on all the switches in order as indicated in this Operating Manual to check that the chair is operating properly.</b></li> <li>● <b>Before using, raise the reclining seat to its fully upright position. Then recline the seat back slowly to the most comfortable position, remembering to start off with a lighter massage.</b> → Starting with a deep massage may hurt a user. Elderly people and persons with weak bones should be especially careful when selecting massage intensity.</li> <li>● <b>For safety, sit down in the chair slowly without turning or twisting the body to ensure that the spinal rollers are placed at either side of the spine.</b></li> <li>● <b>Do not sit, stand, or lean on the leg rest unless you are sitting in or getting up from the chair.</b> → Personal injury or damage to the chair may result if the chair tips over.</li> <li>● <b>If you use the chair after it has not been used for a long time, read this Operating Manual again thoroughly and check if the chair operates properly.</b></li> </ul>
 Prohibited	<ul style="list-style-type: none"> <li>● <b>Do not use the chair in any way not prescribed by the Operating Manual. Do not use the chair together with any other therapeutic device or with an electric blanket.</b> → May result in injury or have a negative effect on health.</li> <li>● <b>Do not move the chair while someone is in it.</b> → Accident or injury may result if the chair tips over.</li> <li>● <b>Do not sit down on the chair with the leg rest raised.</b> → Any sudden, heavy weight on the leg rest could cause damage to the mechanism. Also, accident or injury may result if the chair tips over.</li> <li>● <b>Do not use with anything other than the accessories supplied with this product.</b></li> </ul>

[SAFETY PRECAUTIONS]

## Safety Precautions: During Use

 <b>Warning</b>	
 Prohibited	<ul style="list-style-type: none"> <li>● <b>Do not wrap the power cord or the remote control cord around the Chair, or place the chair on top of either cord. Do not scratch, tear, treat, unduly twist, stretch or bend the power cord or the remote control cord.</b> → The cords may be damaged, resulting in fire or electric shock.</li> <li>● <b>Do not spill water and/or other liquids on the chair or remote control.</b> → May result in electric shock, short circuit, or combustion.</li> <li>● <b>Children and those who are unable to express their intentions should not use the chair. It is advisable to keep children and pets off the chair and away from the surrounding area (especially behind the reclining seat back or under the seat or leg rest).</b> → Could result in accident or injury.</li> <li>● <b>Do not use the chair while more than one person is sitting in it. In particular, do not use the chair while holding a small child (or baby).</b> → Could result in accident or injury.</li> <li>● <b>Do not use the chair while your body is wet, or with wet hands.</b> → Could result in electric shock or damage to the chair.</li> <li>● <b>Do not use the chair if the covering is torn. If it is, unplug the power cord from the wall outlet.</b> → Could result in accident, injury or electric shock. Contact the vendor for repairs.</li> <li>● <b>Be careful not to get hands, feet, or objects caught between the units of the leg rest or between the leg rest and the chair. Be careful not to get hands, feet, or objects caught between the units of the armrest or between the armrest and the chair.</b> → Accident, personal injury or damage to the chair may result if hands, feet or objects are caught.</li> <li>● <b>Do not release hands suddenly while lifting the leg rest by hand.</b></li> </ul>
 Required Action	<ul style="list-style-type: none"> <li>● <b>Make sure no one, especially children or pets, is in the way when reclining the seat back or lowering the leg rest down. Be careful that people and/or objects are not caught behind the seat back or under the leg rest.</b></li> </ul>
 <b>Caution</b>	
 Prohibited	<ul style="list-style-type: none"> <li>● <b>Do not use the chair unclothed.</b></li> <li>● <b>Do not use the chair for purposes other than massage, or other than the ordinary uses of a chair.</b> → Could result in accident or injury.</li> <li>● <b>Do not use the chair to massage the head, chest, stomach, and/or joints (knees, elbows). Do not place hands, feet, or arms in between the rollers.</b> → May result in nausea or injury. Place a soft towel behind your neck if you want to avoid intense stimulation of the area.</li> <li>● <b>Do not place hands or feet into the roller tracks, into the space between the seat back and the seat or into the moving parts. Do not sit wearing hard objects, such as hair accessories or a necklace.</b> → May result in personal injury.</li> <li>● <b>Do not use the neck massage unit anywhere other than behind your neck.</b> → May result in personal injury.</li> <li>● <b>Do not use anything other than legs and feet on the leg rest.</b> → May result in personal injury.</li> <li>● <b>Do not operate the massage chair or leave it operating when there is no one in it.</b> → Could result in accident or injury.</li> </ul>

# SAFETY PRECAUTIONS

## Safety Precautions: During Use

### Caution

 <p>Required Action</p>	<ul style="list-style-type: none"> <li>● In an emergency, or if you find any abnormal condition, push the EMERGENCY STOP button on the remote control to stop operation immediately.</li> <li>● If you get acute pain or other unusual physical discomfort while using the chair, immediately discontinue the use of the chair and seek medical advice.</li> <li>● If you feel the massage is too strong, reduce the stimulation according to this Operating Manual. If this solution does not work, stop using and contact the vendor.             <ul style="list-style-type: none"> <li>→ May result in injury if you continue use with high stimulation under such circumstances.</li> </ul> </li> <li>● Make sure to keep the back pad in place during massage. Do not use the chair without the back pad.             <ul style="list-style-type: none"> <li>→ May result in personal injury.</li> </ul> </li> <li>● Do not fall asleep in the chair during a massage session.             <ul style="list-style-type: none"> <li>→ May result in personal injury.</li> </ul> </li> <li>● Keep massage sessions to no more than 15 minutes in length. Do not exceed five minutes for area specific massage of any particular part of the body.             <ul style="list-style-type: none"> <li>→ Continuous use for extended periods of time can result in over-stimulation, which could cause the opposite of the desired effect, or even injury. The total amount of time for a daily massage should be limited to about 30 minutes keeping appropriate intervals between each massage session.</li> </ul> </li> <li>● If your leg or foot comes out of the leg rest during the massage, do not try to force it back in.             <ul style="list-style-type: none"> <li>→ Undue force may result in damage to the mechanism.</li> </ul> </li> </ul>
 <p>Unplug the chair</p>	<ul style="list-style-type: none"> <li>● If there is a power outage or the possibility of a power outage, stop operation immediately and turn the main POWER switch to OFF. Unplug the power plug from the wall outlet.             <ul style="list-style-type: none"> <li>→ Accident or injury may result if the chair is still in operating mode when the power resumes.</li> </ul> </li> <li>● If there is a threat of lightning, stop operation immediately and turn the main POWER switch to OFF. Unplug the power plug from the wall outlet.</li> <li>● If an operation does not start, or if you find any abnormality, stop the chair immediately and turn the main POWER switch to OFF to prevent an accident. Unplug the power plug from the wall outlet. Contact the vendor for inspection or repairs.             <ul style="list-style-type: none"> <li>→ Electric shock or fire caused by a short circuit may result if the chair is left plugged in under such circumstances.</li> </ul> </li> </ul>

## Safety Precautions: After Use or When Not in Use

### Caution

 <p>Required Action</p>	<ul style="list-style-type: none"> <li>● Be sure to turn the main POWER switch to OFF after every use and to unplug the power plug from the wall outlet.             <ul style="list-style-type: none"> <li>→ Accident or injury may result if children or pets play with the chair.</li> </ul> </li> <li>● After using the chair, be sure to return the reclining seat back to its upright position and lower the leg rest to its lowest position.</li> <li>● Wipe the chair off after every use, and store the chair in a place free of excess humidity.</li> <li>● Do not place any object on the seat, reclining seat back, or leg rest.</li> </ul>
 <p>Unplug the chair</p>	<ul style="list-style-type: none"> <li>● Hold the power plug when unplugging from the wall outlet. Do not pull on the power cord.             <ul style="list-style-type: none"> <li>→ May result in electric shock or fire caused by short-circuiting.</li> </ul> </li> <li>● When cleaning the chair, make sure to unplug the power plug from the wall outlet first. Do not plug or unplug the power plug with wet hands.</li> <li>● If the chair is left unused for a while, be sure to unplug it from the wall outlet.             <ul style="list-style-type: none"> <li>→ Insulation degradation by dirt or moisture may cause electric shock, or fire due to short-circuit.</li> </ul> </li> </ul>

## Other Precautions

### Warning

 <p>Prohibited</p>	<ul style="list-style-type: none"> <li>● Do not stand on the chair. Do not make a major impact or sit on any part of the chair other than the seat.             <ul style="list-style-type: none"> <li>→ Personal injury or damage to the chair may result if the chair tips over.</li> </ul> </li> </ul>
 <p>Do not disassemble</p>	<ul style="list-style-type: none"> <li>● Do not try to repair any malfunction of the chair. Contact the vendor for assistance.</li> <li>● For safety, if the power cord is damaged, contact the vendor for replacement.</li> <li>● Do not modify or remodel the chair.             <ul style="list-style-type: none"> <li>→ May result in fire, electric shock, or injury.</li> </ul> </li> </ul>

### Caution

 <p>Required Action</p>	<ul style="list-style-type: none"> <li>● If you lend, sell, or otherwise transfer the chair to another party, make sure to include this Operating Manual.</li> </ul>
--	--

### Note

- Before disposing of the product or its accessories, consult your local authorities about the proper procedure.

[SAFETY PRECAUTIONS]

# MAIN FEATURES

## Compact with advanced functions New Hybrid Massage Function

Gives a roller massage from the shoulders to the lower back, and an air massage of the buttocks, calves and behind the neck.

Also equipped with a leg stretching function for the calves that is based on a new mechanism.

## Optic sensors provide for Automatic Shiatsu Point Locator Function

Automatically locates shiatsu points which vary from person to person. This allows for an effective massage that tailors itself to your size and shape.

## Light therapy Therapeutic Illumination Function

Equipped with an illumination function on the operating panel and leg rest that flickers slowly. The gentle light gives visual therapy.

## Easy to adjust to the most comfortable position Semi-Interlocked Automatic Reclining Function

The seat back can be reclined to between 114° and 142° from the floor. The leg rest moves in conjunction with the seat back, and can be further adjusted and moved up and down with air.

## Comfortable rocking Rocking Function

Equipped with a rocking function for moving back and forth. Massage can be performed at the same time as cradle-like rocking.

## Enhanced air massage Neck Stretch Function

The neck massage unit stretches the neck with air to the back of the neck. Great for relieving muscles aches behind the neck.

## Enhanced air massage Leg Stretch Function

Equipped with a new function for the calves, with a stretching effect that combines 2 mechanical and air actions to give a completely new type of massage. The massage is performed with a kneading comfort that has never been experienced before.

## Advanced healthcare programs New Human Hands Operation

Neck shiatsu and shoulder massage, which previously could only be performed within a healthcare program, are now available as manual selections.

## Advanced healthcare programs Full-Body Stretch Action

The expert technique of the rollers and the air massage combine with the reclining of the seat to deliver a unique full-body stretch action. Recommended for people who want to stretch their bodies and refresh themselves.

## Relieve any kind of fatigue Various Course Functions

Preset programs allow you to select the optimal program according to your purpose, while the manual selection mode allows you to combine functions to suit your particular needs.

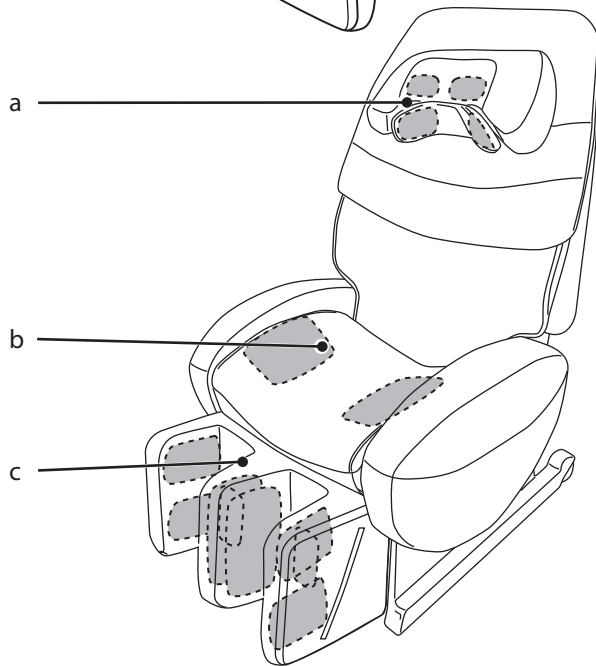
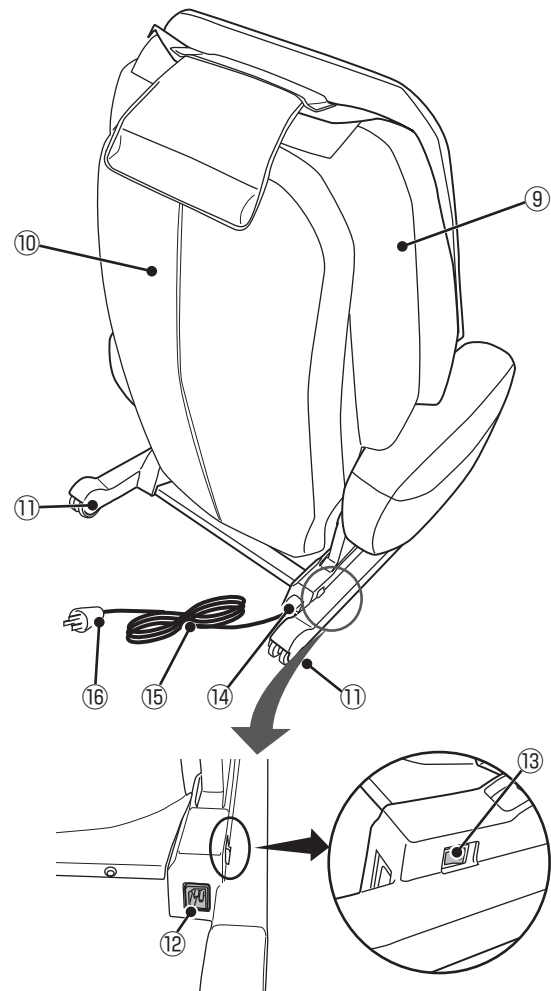
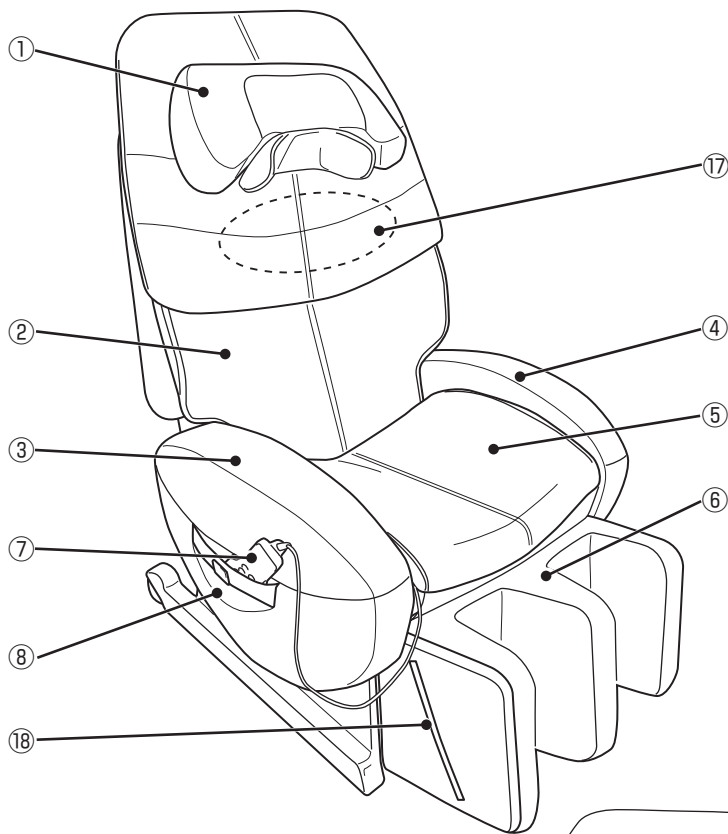
[MAIN FEATURES]

## Benefits

As a therapeutic massage device in the general households, the INADA CHAIR yUMEROBO Rotary Rocker Technology provides the following benefits:

- Relaxes muscles
- Relieves minor muscle aches and pains

# COMPONENTS & DESCRIPTION



[COMPONENTS & DESCRIPTION]

①	Neck Massage Unit
②	Back Pad
③	Armrest (right)
④	Armrest (left)
⑤	Seat
⑥	Leg Rest
⑦	Remote Control
⑧	Side Pocket

⑨	Reclining Seat Back
⑩	Back Cover
⑪	Wheels
⑫	Power Connector
⑬	Main POWER Switch
⑭	Cord Connector
⑮	Power Cord
⑯	Plug

⑰	Rollers	
	Normally stored inside the upper part of the seat back.	
⑱	Leg Rest LED	
a	Neck Air Cushion	x 4
b	Seat Air Cushion	x 2
c	Calf Air Cushion	x 8

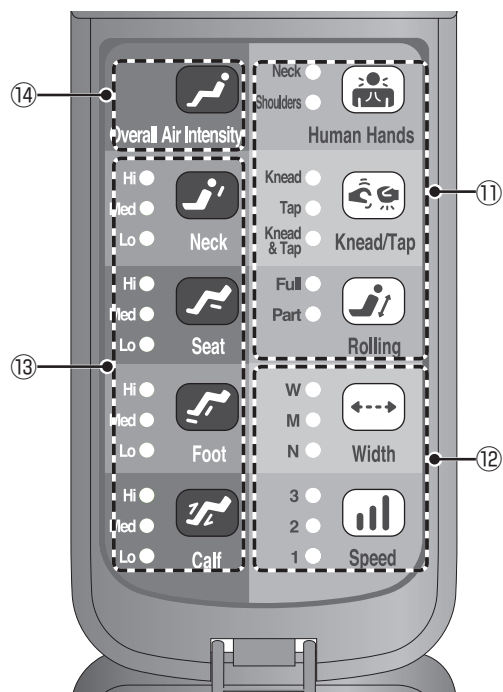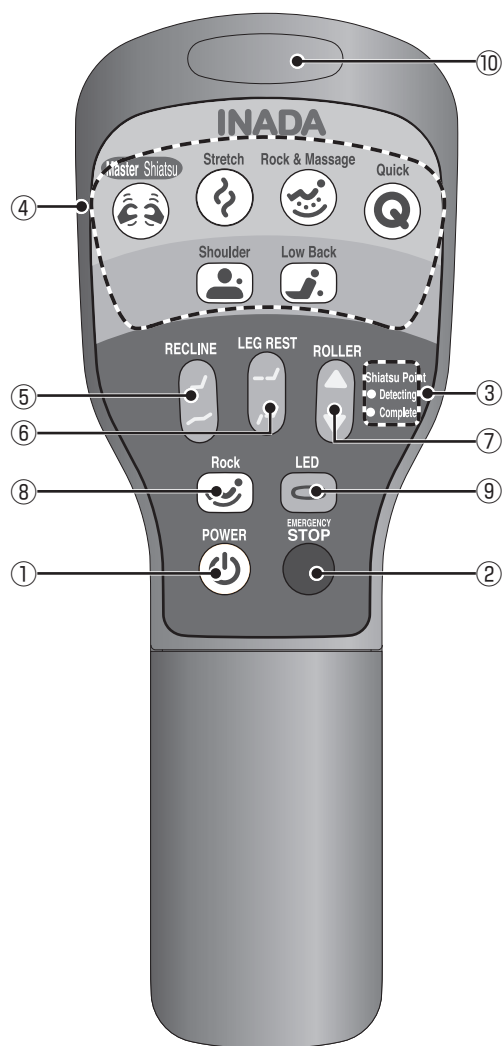1	<b>POWER Button</b>	Turns the remote control power ON and OFF. Pushing the POWER button while the chair is activated causes the rollers to return to the starting position, after which the chair turns itself off. The button lights in green while the power to the remote control is ON, lights in orange while the power is OFF, and flashes in red while the chair is stopped in an emergency.
2	<b>STOP Button</b>	Use this button to shut the chair off immediately in an emergency or if there is a malfunction during operation of the chair. The rollers will stop immediately.
3	<b>Shiatsu Point Locator Indicator</b>	A light indicates when the shiatsu point locator is operating and when the operation has finished.
4	<b>Preset Program Buttons (Healthcare Programs)</b>	Selects the corresponding program.
5	<b>Reclining Button</b>	Use to adjust the angle of the reclining seat back.
6	<b>Footrest Button</b>	Use to adjust the angle of the leg rest.
7	<b>Shoulder Position Adjustment Buttons</b>	Use to adjust the shoulder position while the shiatsu point locator is operating, and to adjust the roller position during manual selection mode.
8	<b>Rocking Button</b>	Use for a rocking operation.
9	<b>LED Button</b>	Selects ON or OFF for the remote control LED and leg rest LED.
10	<b>Remote Control LED</b>	The LED emits light when the LED button is pressed.

11	<b>Basic Operation Buttons</b>	Selects among basic massage operations such as Kneading, Tapping, or Rolling.
12	<b>Intensity Buttons</b>	Adjusts the basic operations.
13	<b>Air Massage Button</b>	Selects among Neck, Seat or Low Body air massage. * Press the buttons repeatedly to adjust the air intensity.
14	<b>Air Intensity Adjust All Button</b>	Adjusts the air intensity of all the air massages that are operating.

⚠ <b>Warning</b>	<ul style="list-style-type: none"> <li>● Do not spill water and/or other liquids on the remote control.</li> <li>● Do not handle the remote control with wet hands.</li> </ul>
⚠ <b>Caution</b>	<ul style="list-style-type: none"> <li>● Do not sit on the remote control. Do not drop or step on the remote control. Do not pull the remote control cord forcibly.</li> <li>● Do not use pointed or hard objects to push the buttons. → May result in damage to the remote control.</li> </ul>

# BEFORE USE

- This product is shipped with the reclining seat back in its fully reclined position. Be sure to raise the seat back to its fully upright position.
- Do not expose the chair to sudden bursts of heat. If the room where the chair is to be used is cold, turn the heat up slowly. If the chair is moved from a cold place into a heated room, wait one hour before using the chair.  
→ Condensation could occur in the moving parts of the chair, which may compromise performance.

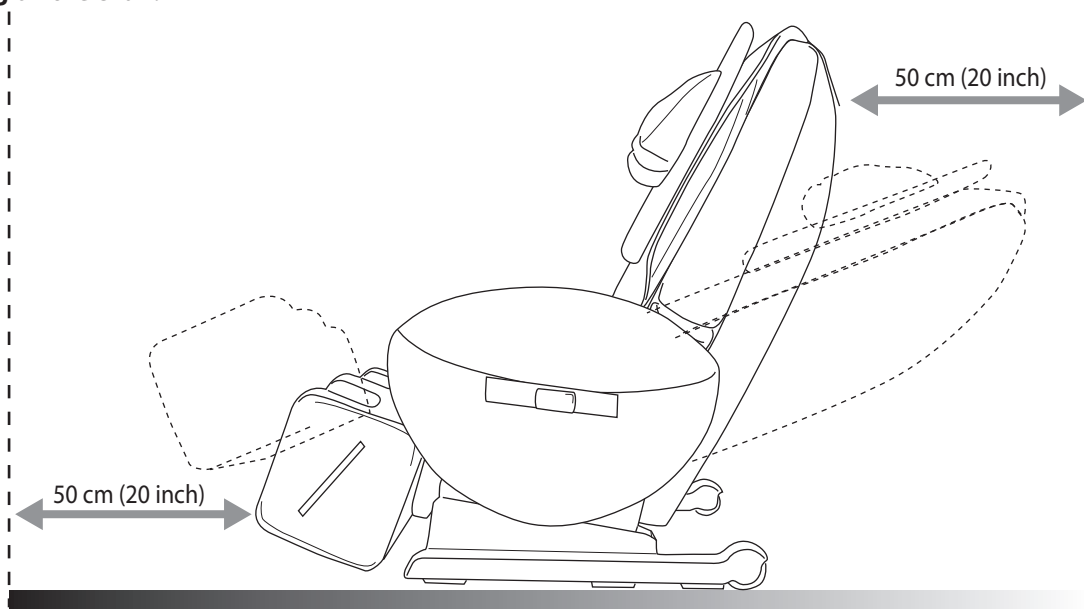
## Installation

Allow for at least 50 cm (20 inch) clearance behind the chair and 50 cm (20 inch) in front of the chair when installing, so as to be able to recline the seat back fully without obstruction.

### ⚠ Caution


- Do not install the chair in an area of high humidity, such as a bathroom or sauna.
- Install on a flat, level surface only.
- Do not use in areas of high debris or dust content. Do not use where ambient temperature is 40°C (140°F) or over.
- Do not install the chair where it can come into contact with direct heat, such as direct sunlight or a radiator.
- Keep fire sources such as cigarettes and ashtrays away from the chair.

- To avoid damaging floor surfaces and to cut down on the noise of the chair, it may be better to install the chair on a carpeted surface or to lay a piece of carpet under the chair. (Do not use an electric blanket.)
- The chair is very heavy, so please be careful when unloading it so as not to damage floor surfaces.
- Install the chair at least 1 m (40 inch) away from TVs, radios, and/or other audio or video equipment. Otherwise they may make noises affected by the operation of the chair.
- The load to the floor per 1 cm<sup>2</sup> (0.16 inch<sup>2</sup>) is approximately 3.0 kg (6.6 lb), with a person weighing 100 kg sitting on the chair.

## Checking the Supplied Items

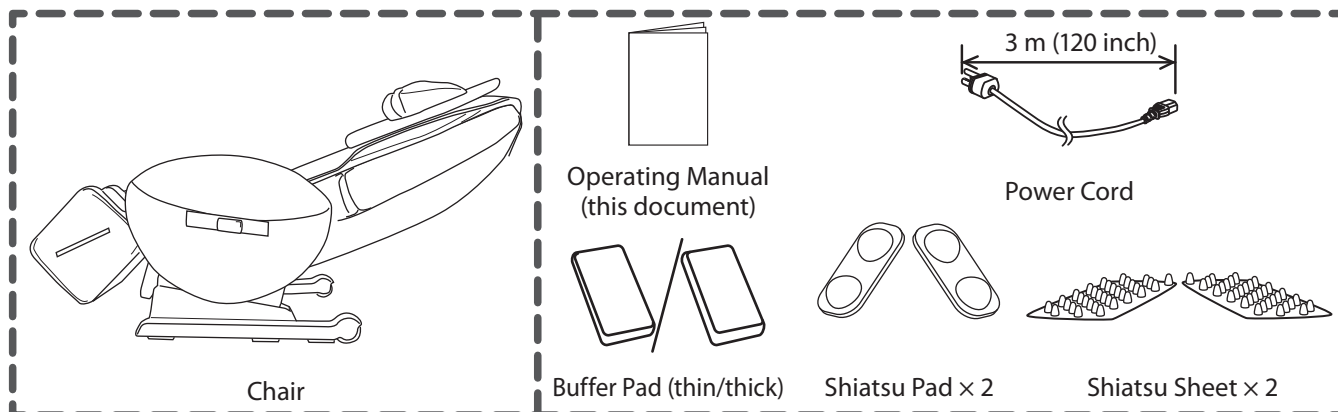Check that the chair and the other items are all in the package.


[BEFORE USE]

### How to Transport or Move the Chair

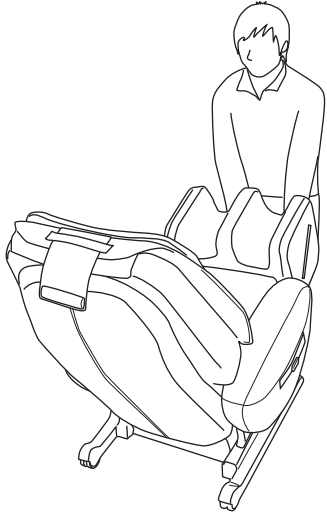Turn the main POWER switch to OFF and disconnect the plug from the wall outlet before moving the chair.


#### To Move with the Wheels

Raise the reclining seat back to its upright position, hold the front of the leg rest, and then move the chair on the wheels by lifting and holding up the front part of the chair.

#### To Carry

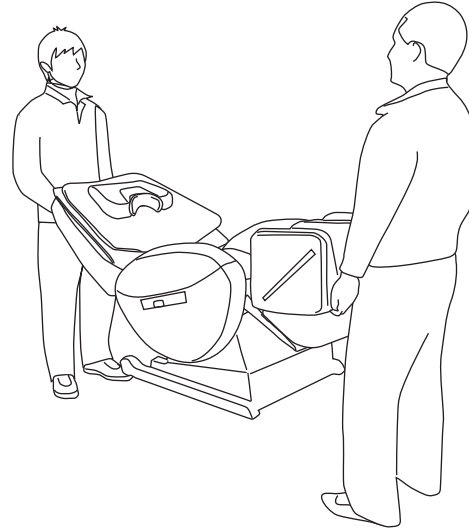Fully recline the seat back and carry the chair by holding the upper part of the seat back and the front of the leg rest.

- The chair is very heavy, so be careful not to hurt your lower back when transporting or moving it.
- The remote control is attached to the chair, so make sure that it does not fall when moving the chair.

#### Caution

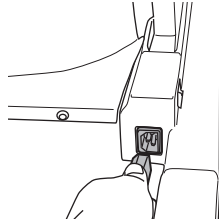
- Do not move the chair while someone is in it.
- Be sure to disconnect the power cord from the wall outlet before moving the chair.  
→ The power cord may be damaged, resulting in fire or electric shock.

[BEFORE USE]


# BEFORE USE

## Main Power Connection

- ① First, plug the power cord into the cord connector located at the rear right of the chair. Then, plug the other end into the wall outlet.

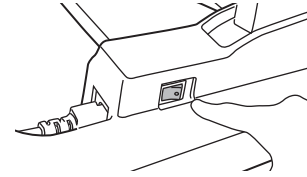- ② Set the main POWER switch located at the rear right of the chair to ON.

\* If the main POWER switch is turned on while the rollers are stopped in the middle of the movement, they will automatically return to the starting position (upper part of the reclining seat) then stop.

### Warning

- **Do not use with any type of transformer device.**  
→ May result in damage to the chair or electric shock.

### Caution

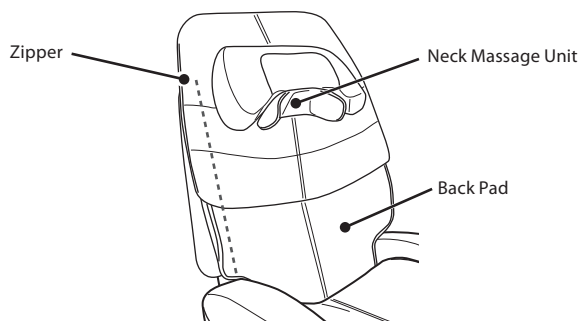
- **Before plugging the power cord into the wall outlet, make sure that the main POWER switch is set to OFF.**
- **Make sure that the plug on the power cord is inserted fully into the wall outlet.**
- **Before using the chair for the first time, turn on all the switches in order as indicated in this Operating Manual to check that the chair is operating properly.**

[BEFORE USE]


## Checking the Seat Cover

Before using the chair, check the seat cover for tearing with the following procedure.

- ① Open the side zipper on the back pad.

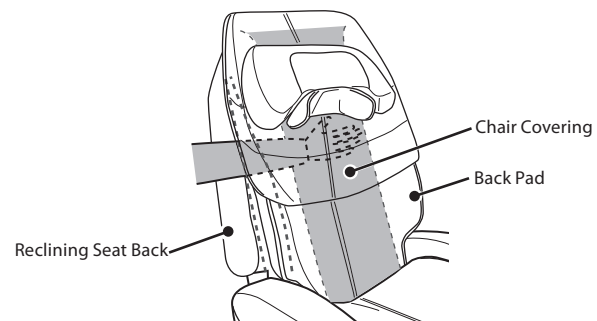- ② Check for holes or tears in the chair covering by inserting your hand between the back pad and the seat back.

### Caution

- **Be sure to check for tearing on the chair before every use, as shown in the illustration. Also, check the other parts for tearing. If you find a tear, no matter how small it is, stop using the chair immediately, unplug the power plug and contact the vendor for assistance. Never take off the chair covering or use the chair while the covering is torn. Doing so is extremely dangerous.**

# POSITION ADJUSTMENT

## How to Sit

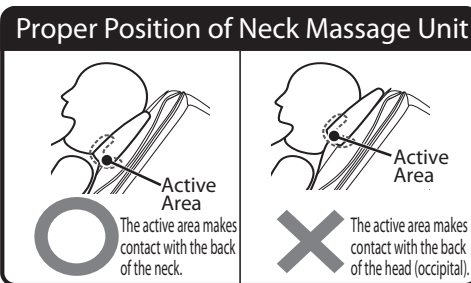
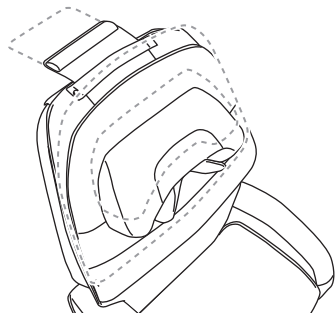
When the chair is off, the rollers are normally drawn back into the left and right sides of the upper part of the seat back (storage position). For the safety, first check the position of the rollers. If they are in any position other than storage positions, turn the main POWER switch on before sitting down to allow the rollers to return to their storage position. When you finish checking, sit down slowly in the center of the seat.

### Caution

- **Do not sit down on the chair with the leg rest raised.**  
→ Any sudden, heavy weight on the leg rest could cause damage to the mechanism. Also, accident or injury may result if the chair tips over.
- **Before using, raise the reclining seat to its fully upright position. Then recline the seat back slowly to the most comfortable position, remembering to start off with a lighter massage.**

## Neck Massage Unit Adjustment

You can adjust the position of the neck massage unit to keep your head in a comfortable, natural position during your massage.

- Because the neck massage unit is balanced by a weight attached to its rear, you can adjust its position simply by moving the neck massage unit up or down. Adjust the neck massage unit to match the height of your head before starting the massage.
- Place the neck massage unit in a proper position. If it is not placed properly, the automatic shiatsu point locator may not function properly.

### Caution

- **Because a weight is attached to the neck massage unit, do not throw the unit or swing it around. Be especially careful if there are small children in your household.**  
→ May result in personal injury.

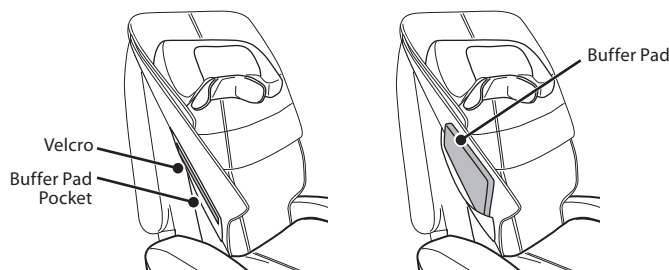
## How to Use the Buffer Pads

If you feel that the massage is too intense, insert the supplied buffer pad (thin) into the rear of the back pad.

If you still feel that the massage is too intense, replace the thin buffer pad with the thick one and start the massage again.


### How to Insert a Buffer Pad

Turn over the back pad, peel open the velcro of the buffer pad pocket on the rear of the back pad, and insert the buffer pad.

For details on how to turn over the back pad, see ① in "Checking the Seat Cover" on page 9.

### How to Properly Use the Buffer Pads
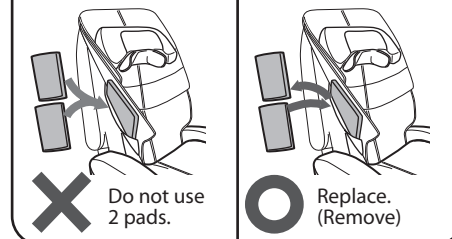
### Caution

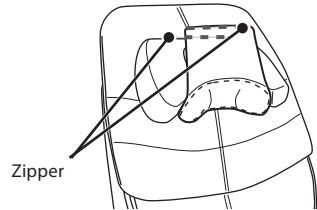
- **Do not use both buffer pads at the same time.**  
→ May result in accident or injury.
- **Make sure that the buffer pad does not curl up or become lopsided.**  
→ May result in accident or injury.

# POSITION ADJUSTMENT


## How to Install and Remove the Cover of the Neck Massage Unit

The chair is shipped with a cover covering the neck massage unit. Remove the cover with the following procedure.

① Open the zipper

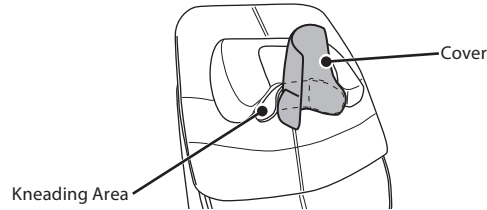② Remove the cover from the kneading area

\*Reattach by following this procedure in reverse.

## How to Use the Shiatsu Pads

Shiatsu pads and shiatsu sheets are supplied with the chair. Use by inserting them into the neck massage unit and the special leg rest pockets.

Shiatsu Pad A × 2

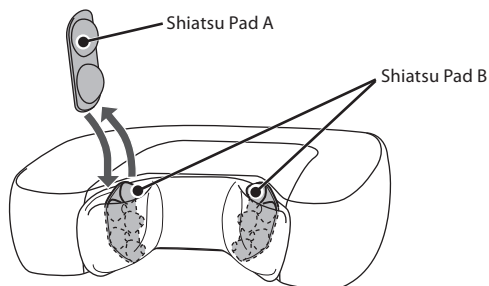
Shiatsu Pad B × 2

Shiatsu Sheet × 2


### Neck Massage Unit

Shiatsu pad B is installed in the neck kneading area of the neck massage unit. If you want, you can replace shiatsu pad B with shiatsu pad A.

### Leg Rest

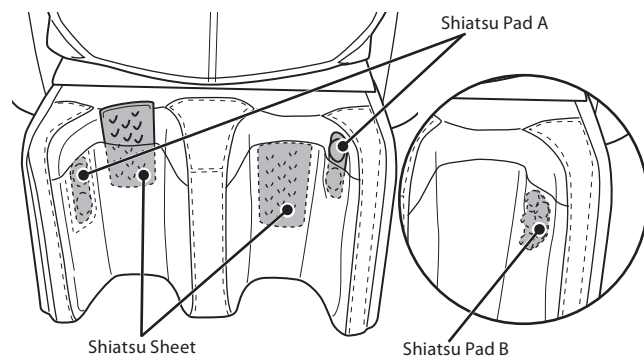Shiatsu pad A, shiatsu pad B and shiatsu sheets can be installed in the leg rest. Replace and use them as you want.


## How to Use the LEDs


When the remote control power is turned on, the remote control LED and leg rest LED turn on. Each time the LED button is pressed, the LEDs switch between on and off.

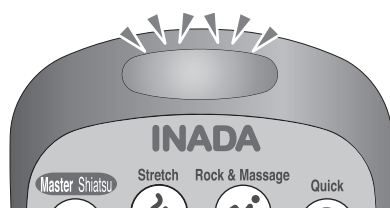Remote Control LED

Leg Rest LED
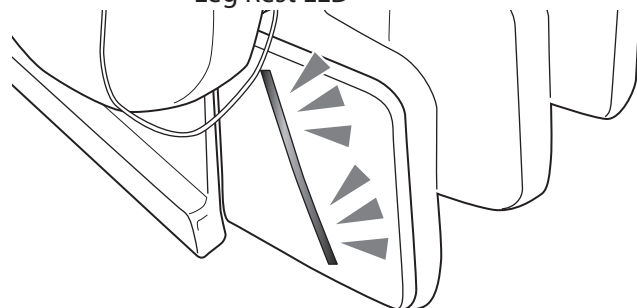
## How to Use the Leg Rest

### Vertical Adjustment

#### ■ To raise the leg rest:

Press the top LEG REST button to raise the leg rest.

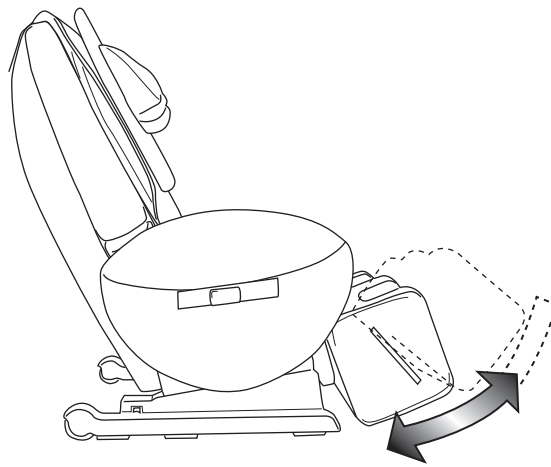
#### ■ To lower the leg rest:

Press the bottom LEG REST button to lower the leg rest.

### Movement Range of Leg Rest

If the leg rest is raised while the seat back is upright, it will only rise to a certain height. To raise to the maximum height, fully recline the seat back.


### Hold Function

Hold the button for about 2 seconds to raise (or lower) the leg rest automatically for a set period of time. If the seat back stops mid-operation, press the button again.

### Leg Rest Extension and Retraction

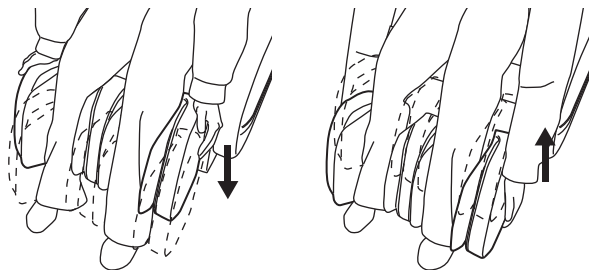When the leg rest is raised, push or pull the leg rest by hand to adjust its position.

### Foot Massage

While the seat back is fully reclined, lift up your knees and place your feet in the leg rest for a foot massage.

#### ⚠ Warning

- Make sure no one, especially children or pets, is in the way when lowering the leg rest. Be careful that people and/or objects are not caught under the leg rest.
- Do not release hands suddenly while lifting the leg rest by hand.

#### ⚠ Caution

- Do not place anything other than legs and feet on the leg rest.

# POSITION ADJUSTMENT

## How to Use the Reclining Seat Back

### ■ To raise the reclining seat back:

Press the top RECLINE button to raise the reclining seat back.

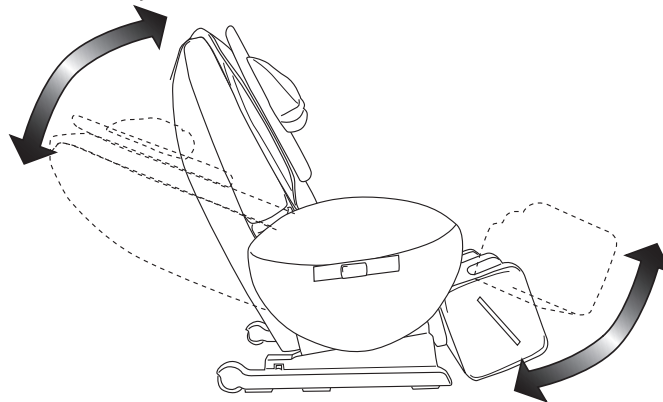
### ■ To lower the reclining seat back:

Press the bottom RECLINE button to lower the reclining seat back.

## Reclining Movement Range

The chair reclines up and down in conjunction with the movement of the seat back and leg rest.

### Hold Function

Hold the button for about 2 seconds to raise (or lower) the reclining seat back automatically for a set period of time\*. If the seat back stops mid-operation, press the button again.

\*The seat back stops automatically when it reaches its maximum position, and the leg rest stops automatically after operating for a certain period of time.

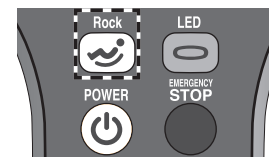
### ⚠ Warning

- Make sure no one, especially children or pets, is in the way when lowering the reclining seat back. Be careful that people and/or objects are not caught behind the reclining seat back.

## How to Rock the Chair

Rocking can be started or canceled in any program and even while the chair is moving.

- \* The rocking lasts about 15 minutes, but if it is stopped mid-operation and restarted, it operates for another 15 minutes. (Note that all operations stop after the rated time of 30 minutes. The rocking operation also stops at this time.)

# AFTER USE

While making sure there is nothing in the surrounding area, return the reclining seat back to its upright position and lower the leg rest to its lowest position.

Set the main POWER switch at the back right of the chair to OFF.

Unplug the power plug from the wall outlet. This will save energy because no standby power will be consumed.

### ⚠ Caution

- Be sure to turn the main POWER switch to OFF after every use and to unplug the power plug from the wall outlet.  
→ Accident or injury may result if children or pets play with the chair.
- Arrange the power cord away from a working area and where it will not be tripped over.
- After using the chair, be sure to return the reclining seat back to its upright position and lower the leg rest to its lowest position.
- Wipe the chair off after every use, and store the chair in a place with low humidity.
- Hold the power plug when unplugging from the wall outlet. Do not pull on the power cord.
- If the chair is left unused for a while, be sure to unplug it from the wall outlet.
- Be careful not to tangle yourself in the cords and trip over.

# PROGRAMS & SELECTION

## PRESET PROGRAMS

The preset programs make it easy to select a program that suits your preferences or objectives.

### Healthcare Programs

Programs that include massages for recovering from muscle fatigue.

#### Master Shiatsu

This program gives a full-body massage using expert techniques.

#### Stretch

This program includes light stretch actions where the entire body is bent backward.

#### Rock & Massage

A program that massages while rocking the chair back and forth.

#### Quick

This program condenses all operations to give a massage in a short time.

#### Shoulder

This program gives a concentrated massage of the neck and shoulders in a short time.

#### Low Back

This program gives a concentrated massage of the lower back in a short time.

## MANUAL SELECTION

You can combine your favorite massage types. You can also adjust the speed, intensity, and roller positions.

# STARTING A MASSAGE

## ① Turn on the power

Press the POWER button to turn on the power.  
(The POWER button lights in green.)

\* If no selection is made within 5 minutes, the power is automatically shut off.

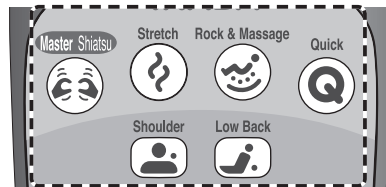
## ② Select a program

### Preset Programs selected

#### Preset Program Messages

- 6 automatic programs provide a variety of different massages to relieve fatigue.

The reclining will start after a short time.  
Then, the shiatsu point locator indicator turns on and shiatsu point locating is performed.

[Note]

Sit as far back as possible in the chair and put your head against the headrest in a relaxed manner. Shiatsu points cannot be accurately located if your body is raised.

### Manual Selection Mode selected

#### Manual Selection Messages

- You can perform massages of your own choice by combining basic operations and air pressure.

Select your operation of choice from a human hand, Knead/Tap, rolling, or air massage.

If you select the human hands, rolling or neck air operation, the shiatsu point locator indicator turns on and shiatsu point locating is performed.

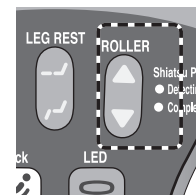
[STARTING A MASSAGE]

## ③ Adjust the shoulder positions

In manual selection mode, when HUMAN HANDS, ROLLING or NECK AIR is selected

After shiatsu point locating is performed, the shoulder positions can be adjusted while the locating indicator is flashing. If no adjustment is required, do not use these buttons.


#### ④ The program starts

##### Preset Programs starts

###### Adjustment in Preset Programs

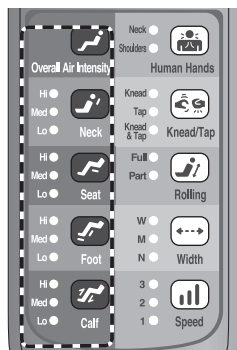
- The air can be canceled or its intensity adjusted.

##### Manual Selection Mode starts

###### Adjustment in Manual Selection Mode

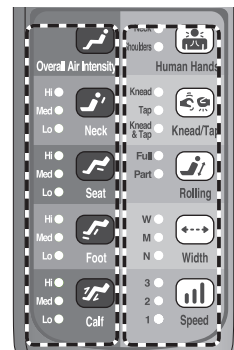- A basic operation can be changed and the air can be canceled or its intensity adjusted.

When shiatsu point locating is completed, the Complete light turns on and the program starts.

#### ⑤ The program finishes

The rollers will return to their storage positions (upper part of the reclining seat back), and the power of the operating mechanism will shut off. (The POWER button flashes while the rollers are returning to their storage positions.)

If you want to raise the reclining seat back, press the POWER button again to turn the power on, and then raise the seat back.

##### Changing Courses

Before the course in progress finishes, you can change it to another course. After a total duration of 30 minutes, the course in progress stops even if not completed, and the POWER button turns orange.

##### Switching from One Preset Program in Progress to Another

When one preset program button is pressed while another preset program is in progress, the rollers return to their storage positions, then the selected program starts from the automatic shiatsu point location operation.

##### Switching from a Preset Program to Manual Selection Mode

When one of the buttons for manual selection mode located inside the panel of the remote control is pressed while a preset program is in progress, the preset program immediately stops, and the selected operation starts. If ROLLING, HUMAN HANDS or NECK AIR is selected, the operation starts after the shiatsu points are located.

(To select an air massage, first select an operation such as kneading or tapping, or press the POWER button to stop the preset program in progress before making the selection.)

##### Stopping in the Middle of a Massage

To stop mid-massage, press the POWER button. The rollers will return to their storage positions and stop.

**In an emergency or if there is some malfunction** during the operation of the chair, press the STOP button. The rollers will stop immediately. After confirming safety, press the POWER button. The rollers will return to their storage positions and stop.

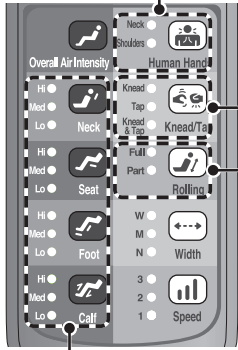
##### Precautions for Preset Programs

When a preset program is selected, the reclining seat back may move automatically. Also, the reclining seat back may move automatically even while a program is in progress. For this reason, before using the chair, make sure that there is no person or obstruction behind it. Also, do not let other people approach the chair during use.

##### Caution

- If you feel the massage is too strong, reduce the stimulation according to this Operating Manual. If this solution does not work, stop using and contact the vendor.  
→ May result in injury if you continue use with high stimulation under such circumstances.
- Do not place hands or feet into the roller tracks, into the space between the seat back and the seat or into the moving parts. Do not sit wearing hard objects, such as hair accessories, or a necklace, watch, bracelet or ring.
- If your leg or foot comes out of the leg rest during the massage, do not try to force it back in.
- Keep massage sessions to no more than 15 minutes in length. Do not exceed 5 minutes for area specific massage of any particular part of the body.  
→ Continuous use for extended periods of time can result in over-stimulation, which could cause the opposite of the desired effect, or even injury. The total amount of time for a daily massage should be limited to about 30 minutes.

# MASSAGE ADJUSTMENTS

**Human Hands**  
Each time the button is pressed, switches between Neck Shiatsu, Shoulders, and Off.

Operation	Action	Speed Adjustment	Width Adjustment	Up/Down Adjustment
Neck	This performs shiatsu to the neck.	—	—	—
Shoulders	This performs a shoulder kneading massage.	—	—	—

\* Before selecting neck shiatsu, turn over the neck massage unit to the rear.

**Knead/Tap**  
Each time the button is pressed, switches between Kneading, Tapping, Kneading & Tapping, and Off.

Operation	Action	Speed Adjustment	Width Adjustment	Up/Down Adjustment
Knead	This performs a kneading type of massage.	○	—	○
Tap	This performs a tapping type of massage.	○	○	○
Knead & Tap	Kneading and tapping are simultaneously performed.	○	—	○

**Rolling**  
Each time the button is pressed, switches between Full, Partial, and Off.

Operation	Action	Width Adjustment	Up/Down Adjustment	
Rolling	This performs full or partial stretching of the spine using up and down movements.	Full Rolling	○	○
	Part Rolling	○	○	

**Air Massage**  
Each time the button is pressed, the intensity switches between Low, Medium, High and Off.

Operation	Action	Intensity Adjustment
Neck	Air massage to the neck. (Shiatsu point locating is performed.)	○
Seat	Air massage to the seat.	○
Foot	Air massage to the legs that feels like kneading.	○
Calf	Air massage to the legs that feels like stretching.	○

[MASSAGE ADJUSTMENTS]

Operation Combination List for Manual Selection Mode

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
- Can be operated at the same time
- × Ends the current operation and changes to the selected operation
- Cannot be selected


## Various Adjustments
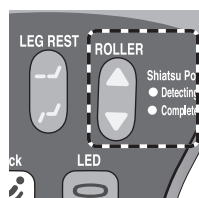
### Shoulder Position Adjustment


If the correct shoulder position is not set during shiatsu position search, adjust the shoulder position while the locator indicator is flashing.

▲	The rollers move up while this button is pressed.
▼	The rollers move down while this button is pressed.
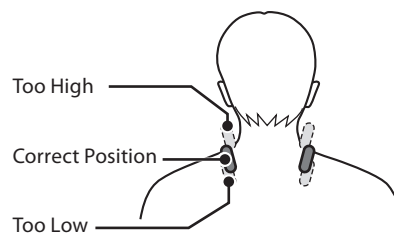
### Roller Position Adjustment

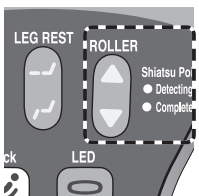When kneading or tapping is selected:  
The roller position can be adjusted to the area that you want to massage.

▲	The rollers move up while this button is pressed.
▼	The rollers move down while this button is pressed.

When full rolling is selected:  
Changes between up and down.

▲	When pressed during a down movement, changes to up.
▼	When pressed during an up movement, changes to down.

When partial rolling is selected:  
The center position of the range of partial rolling can be adjusted.

▲	The rollers move up while this button is pressed.
▼	The rollers move down while this button is pressed.

When neck air or human hands kneading is selected:

× The roller position cannot be adjusted

● When up (or down) is selected when the roller is already at the highest (or lowest) position, a warning alarm sounds.

### Caution!

● If neither a basic operation nor rolling has been selected, there will be no operation even if you press a shoulder position adjustment button.

**Intensity Adjustment for Air Massage ① Adjust all**  
The intensity of all areas can be adjusted to the same level. Each time the button is pressed, the intensity switches between Low, Medium, High, and Off.  
**Caution!**  
The operations cannot be stopped.

**Intensity Adjustment for Air Massage ② Adjust individually**  
The intensity can be adjusted for each area. Each time a button is pressed, the intensity changes in the order of Low, Medium, High and Off.

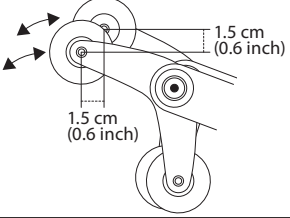
**Width Adjustment**  
During a basic operation where width adjustment is possible, the width of the rollers can be adjusted in 3 steps (narrow, medium and wide).

**Speed Adjustment**  
During a basic operation where speed adjustment is possible, the speed of action can be adjusted in 3 steps.

[MESSAGE ADJUSTMENTS]

# TROUBLESHOOTING

If there is still a problem after trying the solutions given in this troubleshooting guide, stop using the chair immediately to prevent accident or injury, and unplug the power plug from the wall outlet. Contact the vendor or your nearest service center for assistance.

Problem	Cause & Solution	
The right and left rollers are out of alignment.	<p>The rollers are designed with an alternating tapping mechanism, so in the stopped position they are sometimes out of line. This is part of the design and is not a malfunction.</p> 	This is part of the design and is not a malfunction.
When the reclining button is pressed, sometimes the seat back starts moving in the opposite direction once.	This is part of the design and is not a malfunction.	
The rollers moved up to the top and then stopped.	The auto timer will cause the rollers to move up to the top of the chair when the massage time is finished, so as not to interfere with your movements when getting out of the chair.	
The rollers stop during the massage.	Sometimes when there is too much pressure to the rollers, they can stop moving temporarily. Slightly lift your body or lightly raise the reclining seat back. (Be especially careful when a person who weighs 100 kg or more uses this chair.)	If there is still a problem after the check, stop using the product immediately.
The POWER button of the remote control and the program button flash.	Set the main POWER switch located at the front right of the chair to OFF then back ON after about 5 seconds. →See P9	
The automatic shiatsu point locator function is not working properly. (The height of the rollers does not match the shoulders.)	Sit further back in the chair so that your head rests firmly on the neck massage unit. You can adjust the shiatsu position with the ROLLER buttons immediately after the shiatsu points are located. →See P10, 16	
Remote control buttons do not seem to be operating normally. Or the power does not turn on.	<p>Check for the following:</p> <ol style="list-style-type: none"> <li>① Is the power cord fully plugged into the wall outlet? →See P9</li> <li>② Is the power switch located on the right side of the chair set to ON? →See P9</li> <li>③ Are you pressing 2 buttons at the same time?</li> </ol>	
The power cord or plug is abnormally hot.	Stop using immediately.	
Contact the vendor or your nearest service center.		
Anyone other than an authorized serviceperson must not repair or disassemble.		

[TROUBLESHOOTING]

Noise during chair operation

These noises are a normal part of the chair's operation and do not affect its performance in any way.

- The following noises are made during operation.
- |   |
|---|
| Noises during tapping   |
| Sound of the rollers rubbing against the seat cover during kneading   |
| Sound of raising and lowering the rollers, sound of roller operations |
| Sound of the air massage cushions                                     |
| Sound of the air pump   |
| Sound of air discharge  |
| Sound of raising and lowering the reclining seat back                 |

## REGULAR MAINTENANCE FOR SAFETY

### Regular Maintenance Will Preserve the Life of Your Chair

#### Have you experienced any of these problems?

- The chair does not start with the Main POWER switch set to ON.
- The power cord has been cracked or scratched.
- The power cord or plug becomes abnormally hot.
- The chair produces a burnt smell, abnormal sound or vibration.
- The auto timer function does not work in some cases.
- Other failures.

If you find anything listed on the left, stop using and unplug the power cord from the outlet to prevent failure or an accident. Be sure to contact the vendor or our service center for maintenance.

#### To use safely

**To use safely**  
The chair fabric (roller fabric) is a consumable part, and its service life is 3 years (conditions of use: two 15-minute sessions a day, every day).  
For safe use, periodic inspection by the vendor is recommended before the end of service life.  
\*Durable years vary depending on conditions of use.  
\*After about 5 years of use, a professional inspection is recommended even if there are no symptoms or problems.  
For inspection fees, contact the vendor or our service center.

[REGULAR MAINTENANCE FOR SAFETY]  
[TROUBLESHOOTING]

# FAQ

**Q** Can I put the chair onto an electric carpet?

**A** Do not place the chair onto an electric carpet because it could cause a fire.

The chair is very heavy, so it could damage the heat generating parts of the electric carpet and start a fire. → See P7

**Q** Can I put the chair onto a wooden floor or a heated floor?

**A** Yes.

However, because the chair could damage the floor surface, we recommend laying a carpet or something similar under the chair. (Same applies to other floor surfaces) → See P7

**Q** How do I make the massage deeper?

**A** Perform the following:

- Sit back further in the chair so that your weight is against the reclining seat back.
- Lower the reclining seat back. → See P13
- Remove the buffer pad from behind the back pad.

(When installed) → See P10

With the methods above, you can intensify the massage.

**Q** How do I make the massage lighter?

**A** Perform the following:

- Raise the reclining seat back. → See P13
- Insert the thin or thick buffer pad into the rear of the back pad. → See P10
- Remove the shiatsu pad and shiatsu sheet.

(When installed) → See P11

With the methods above, you can make the massage more gentle.

**Q** Can I use both buffer pads at the same time?

**A** Do not use both pads at the same time.

Using both buffer pads at the same time may injure your body.

**Q** Does the reclining seat back move automatically during a massage or after a program has finished?

**A** The reclining seat back does not move automatically after a program has finished.

However, depending on the selected program, the reclining seat back may move automatically while a program is in progress.

**Q** Can someone who is under 150 cm or over 180 cm use the chair?

**A** Yes.

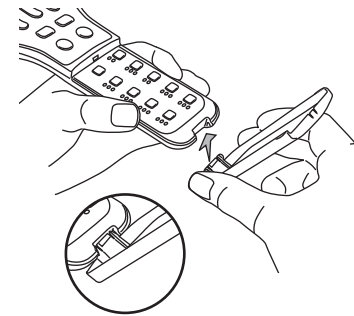
Because the correct shoulder positions may not be found during shiatsu point locating, we recommend using the chair as follows.

- Short person: Raise the reclining seat back → See P13
- Tall person: Recline the reclining seat back → See P13
- After shiatsu point locating is performed, adjust the shoulder positions. → See P18

**Q** The remote control panel has come off. Can I fix it?

**A** Reattach with the following method.

Hold the remote control as illustrated, and reattach by inserting the projection on the bottom of the panel into the hole on the remote control.

\*You may damage the remote control panel if you open it too much. If you damage the panel, contact the vendor.

**Q** I spilled water on the chair. What should I do?

**A** There is a danger of an electric shock. Immediately stop using the chair and perform the following.

- Set the main POWER switch located at the front right of the chair to OFF.
- Unplug the power plug from the wall outlet.

After performing the above, contact the vendor for assistance.

# CARE AND STORAGE

## Storage

- After wiping off any dust or dirt, store the product in a place that is relatively dry.
- If you are not going to use the product for some time, put a cover on it that will protect it from dust and dirt.
- Pets can sometimes chew on power cords, so try to keep pets away from the product even when it is being stored.

### Caution

- **Do not install the product where it can come into contact with direct heat, such as direct sunlight or a radiator.**

## Care and Maintenance

### Caution

- **When cleaning or caring for the product, make sure to unplug the product from the wall outlet first. Do not plug or unplug the power cord with wet hands.**

### Caring for the Back Pad and Covers

Soak a piece of soft cloth in lukewarm water in which a small amount of mild detergent has been added. Then thoroughly wring it out before cleaning off any spots or grime.

### Caution

- **The fabric color can change if the product is in contact with vinyl covers or some wall coverings for a prolonged period of time.**

### Caring for the Remote Control

Wipe dirt off the remote control with a dry cloth.

- Never use a moistened or wet cloth. → May result in mechanical failure.

### Caring for Other Plastic Parts

Soak a piece of soft cloth in lukewarm water in which a small amount of mild detergent has been added, thoroughly wring it out, and clean off any spots or grime. Then wipe with a dry cloth so that no detergent residue remains.

- Do not use alcohol based or other household cleaning products or abrasives because they can scratch or crack the finish and/or cause color to fade.

# CUSTOMER SERVICE AND WARRANTY

## 1. Warranty

Please read the contents carefully and keep it in a safe place. Please complete and mail in the warranty card that was included with your chair.

If the chair is to be used outside the home (in a commercial setting, for example), fees will be charged for repairs.

## 2. Repairs & Service

Before calling for repairs or service, use the Troubleshooting guide on pages 19-20 to see if the problem can be solved easily.

If the problem persists, contact the vendor for assistance. Have the following information ready.

- 1) Your name, address, and phone number
- 2) Product name and model number (see the left rear of the chair)
- 3) Serial No. (see the Serial No. label on the the left rear of the chair)
- 4) Date of purchase
- 5) Trouble status
- 6) Proof of purchase

- **If the warranty is valid:**

A service charge may apply, subject to the nature of the trouble and/or repair according to the terms of the warranty. See the warranty for full details.

- **If the warranty has expired:**

See the back of this booklet and contact our technical service department.

If the product can be repaired, we will give you an estimate of the cost to repair the product.

- **Replacement parts stock:**

We will stock service parts (except fabric) of this product for at least six years after terminating the production of this model. Service parts refer to those necessary for maintaining the intended performance of the product.

- **For more information on service:**

Please see the back page of this booklet.

# SPECIFICATIONS

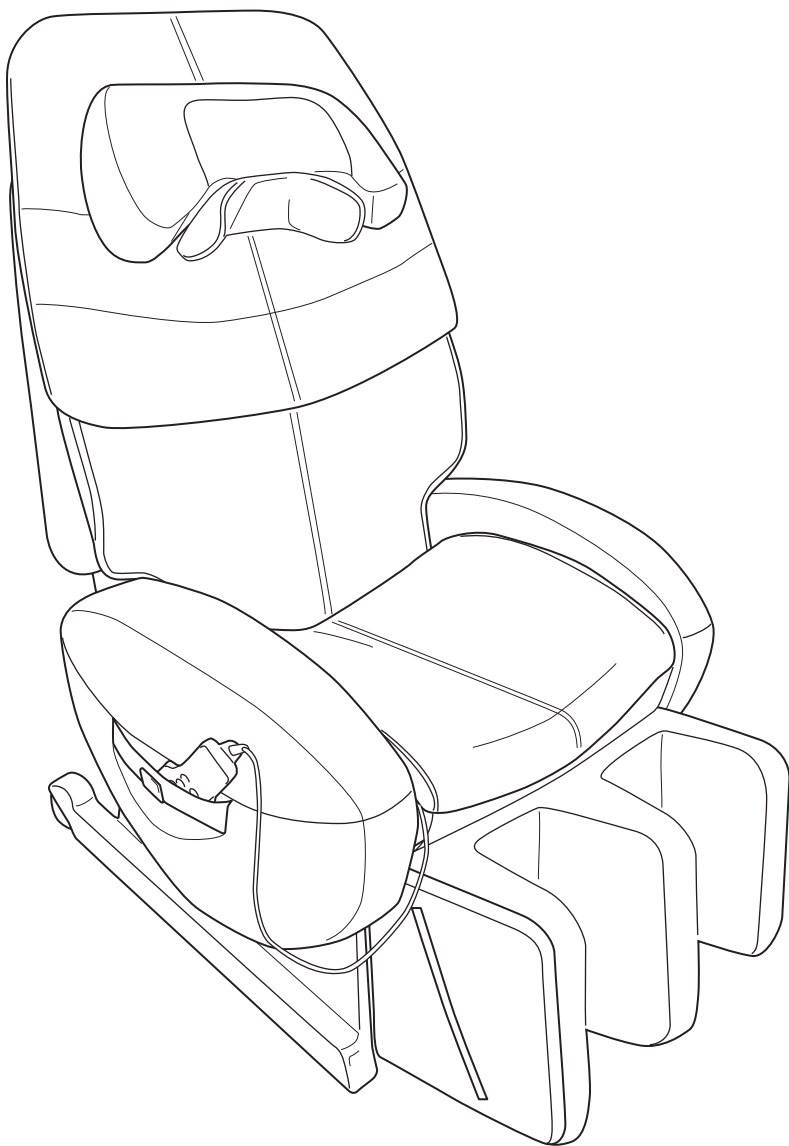
Model Number	HCP-R100D	
Power Supply	AC 220 V 50 Hz AC 220-240 V 50-60 Hz	
Power Consumption	140 W	
Rated Time	30 minutes	
Upper Body	Kneading Speed	3 levels, from minimum 10 times/minute to maximum 32 times/minute (approx.)
	Tapping Speed	3 levels, from minimum 180 times/minute to maximum 500 times/minute (approx.)
	Roller Width	3 steps (narrow, medium and wide)
	Up/Down Speed	One full cycle in approx. 25 seconds (approx. 4.1 cm/sec (1.6 inch/sec))
	Back Stroke Range	Approx. 66 cm (26 inch) (in partial roller massage: approx. 15 cm (6 inch))
Rocking Frequency	5±2 times/minute	
Stretching Frequency	From minimum 12±2 times/minute to maximum 33±2 times/minute (approx.)	
PRESET PROGRAMS	Healthcare programs: 6 types	
MANUAL SELECTION	11 types (automatic shiatsu point locator function included in some programs)	
Air Pressure	60 Hz: Approx. 37 kPa, 50 Hz: Approx. 31 kPa	
Auto Timer	Approx. 15 minutes (preset programs vary in duration)	
Leg Rest Adjustment Range	Approx. 7 cm (2.8 inch)	
Reclining Angle	Approx. 114° - 142° from floor level, continuous motion (Automatic reclining)	
Rocking Operation Range	Seat back angle of approx. 142° to 157°	
Chair Dimensions	Approx. 74 cm × 121 (160) cm × 109 (77) cm (W/D/H) (Approx. 29 inch × 48 (63) inch × 43 (30) inch (W/D/H)) <small>The depth and height figures in ( ) represent the dimensions when fully reclined and with the leg rest flat</small>	
Weight	Approx. 80 kg (176 lb)	
Outer Covering	Synthetic leather	
Supplied Items	Power cord, buffer pad (thick)/(thin), shiatsu pad A × 2, shiatsu pad B × 2, shiatsu sheet × 2	

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Manufactured by FAMILY CO., LTD.