CONTENTS

SAFETY PRECAUTIONS	3	
MAIN FEATURES & BENEFITS	6	
COMPONENTS & DESCRIPTION	7	
BEFORE USE.....	9	
POSITION ADJUSTMENT	13	
AFTER USE.....	15	
PROGRAMS & SELECTION.....	16	
PRESET PROGRAMS	17	
MANUAL SELECTION.....	19	
TROUBLESHOOTING.....	22	
REGULAR MAINTENANCE FOR SAFETY	23	
FAQ	24	
CARE AND STORAGE	25	
AFTER SALES SERVICE.....	26	
SPECIFICATIONS	27	

- Please read this Operating Manual before use.
- Keep this Operating Manual nearby so that you can refer to it anytime.
- This product is designed for home use.

SAFETY PRECAUTIONS

Be sure that you have read these SAFETY PRECAUTIONS thoroughly before using your INADA chair for the first time. The Operating Manual contains important information about how to use the chair safely and correctly, avoiding possible damage to property or injury to persons using the chair and others. First of all, you should understand well the meaning of the indications and symbols described below, which are used throughout the Operating Manual to highlight important safety instructions. Please adhere to all precautions. Descriptions of each chair component can be found on pages 7-8.

Types of Indications			
	Warning This action could result in serious injury or death.		
	Caution This action could result in personal injury or damage to property.		
Types of Symbols and Description			
	Prohibited.		Do not disassemble.
	Do not handle with wet hands.		Do not use in a location where the chair may become wet.
	Do not use in a bathroom.		
	Required action.		Unplug the power plug from the wall outlet.

Remember to keep this Operating Manual in an easily accessible place.

Safety Precautions: Installation			
	<ul style="list-style-type: none"> Do not install the chair in an area of high humidity, such as a bathroom or sauna. → Could result in electric shock or damage to the chair. 		<ul style="list-style-type: none"> Never connect the ground wire to gas pipes, water pipes, telephone lines, or lightning rods.
	<ul style="list-style-type: none"> Install on a flat, level surface only. → Chair could tip over accidentally if placed on an uneven surface, resulting in an accident. 		

Safety Precautions: Installation		
	<ul style="list-style-type: none"> Space Requirements This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electric current in order to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances. • Improper connection of the equipment-grounding conductor can result in the risk of electric shock. Check with a qualified electrician or serviceman if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product if it will not fit the outlet; have a proper outlet installed by a qualified electrician. This product uses on a normal Alternating Current(AC) circuit and has a grounding plug that looks like the plug illustrated in the figure to the right. Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product. 	
	<ul style="list-style-type: none"> Always unplug this appliance from the electrical outlet immediately after using and before cleaning. 	

	<ul style="list-style-type: none"> Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons. Use this appliance only for its intended use as described in this Operating Manual. Do not use attachments not recommended by the manufacturer. Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair. Do not carry this appliance by supply cord or use cord as a handle. Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like. Never drop or insert any object into any opening. Do not use outdoors. Do not operate where aerosol (spray) products are being used or where oxygen is being administered. Children should be supervised to ensure that they do not play with the appliance.
	<ul style="list-style-type: none"> An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts. Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons. Keep the cord away from heated surfaces. To disconnect, turn all controls to the off position, then remove plug from outlet. Keep children away from extended foot support (or other similar parts). Connect this appliance to a properly grounded outlet only. See Grounding Instructions. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.

Safety Precautions: Before Use

Warning

 Prohibited	<ul style="list-style-type: none"> ● The following persons must not use this massage chair: <ul style="list-style-type: none"> ○ Persons who have been instructed by a doctor to avoid massage therapy. Ex. Thrombosis, severe aneurism, acute varicosity, or dermatitis and/or skin infection including inflammation of the hypodermis. ○ Persons with suspected acute neck sprain (whiplash injury) ● Do not use a power supply other than the one specified. <ul style="list-style-type: none"> → May result in fire or electric shock. ● Do not use with any type of transformer device. <ul style="list-style-type: none"> → May result in damage to the chair or electric shock. ● Do not use the chair if the power cord or plug is damaged, or if the wall outlet connection is loose. <ul style="list-style-type: none"> → Persons result in electric shock, short circuit, or combustion. ● Before each use, check for holes or tears in the fabric of the active area. Also, check the other parts for tearing. Unplug the chair and contact your local distributor for assistance. Never use the chair with the fabric of the active area removed or with holes or tears in the fabric, because doing so is extremely dangerous. <ul style="list-style-type: none"> → Using the chair with holes or tears in the fabric could result in injury or electric shock. 	<table border="1"> <thead> <tr> <th>Model number</th> <th>Power supply</th> </tr> </thead> <tbody> <tr> <td>HCP-WG1000A</td> <td>AC120V~</td> </tr> <tr> <td>HCP-WG1000D</td> <td>AC220-240V~</td> </tr> <tr> <td>HCP-WG1000E</td> <td>AC110V~</td> </tr> </tbody> </table>	Model number	Power supply	HCP-WG1000A	AC120V~	HCP-WG1000D	AC220-240V~	HCP-WG1000E	AC110V~
Model number	Power supply									
HCP-WG1000A	AC120V~									
HCP-WG1000D	AC220-240V~									
HCP-WG1000E	AC110V~									
 Required Action	<ul style="list-style-type: none"> ● Persons with any of the following conditions should seek medical advice before using the chair: <ul style="list-style-type: none"> ○ Serious heart conditions ○ Sensory impairment caused by serious peripheral circulatory disorders due to diabetes and other diseases ○ Osteoporosis, spinal fractures, acute pain and/or symptoms thereof ○ Persons being treated for personal injuries ○ High fever (38°C (100°F) or higher) Ex. Strong acute inflammation illness caused by fatigue, chills, blood pressure fluctuation, or asthenia ○ Persons in the early stages of pregnancy, or immediately after giving birth ○ Persons who have electronic medical device(s) easily susceptible to electromagnetic interference Ex. pacemaker ○ Abnormal or curved spine ○ Persons planning to use the chair for treatment or for rehabilitation purposes ○ Persons with malignant tumor(s) ○ Persons under bed rest orders ○ Persons with acute gastro-intestinal complaints or ailments such as gastritis, enteritis, or hepatitis ○ Persons under a doctor's care, or those experiencing unusual physical discomfort ○ Joint dysfunction due to rheumatism, gout, or other diseases ○ Persons with symptoms such as qualm, dizziness, or tinnitus ○ Inflammation due to sprains, muscle strain, or other injuries ○ Persons with thecitis or suspected thecitis ○ Persons with thermoanesthesia ● Elderly people and persons with weak bones should seek medical advice before using the chair, even if they do not have a specific disease. ● If your condition does not improve after use, consult a doctor or specialist. 									

Caution

 Prohibited	<ul style="list-style-type: none"> ● Do not use the chair in any way not prescribed by the Operating Manual. Do not use the chair together with any other therapeutic device or with an electric blanket. <ul style="list-style-type: none"> → May result in injury or have a negative effect on health. ● Do not move the chair while someone is in it. <ul style="list-style-type: none"> → Accident or injury may result if the chair tips over. ● Do not sit down on the chair while the seat back is reclined. <ul style="list-style-type: none"> → Any sudden impact on the leg rest could cause damage to the mechanism. Also, an accident or injury may result if the chair tips over. ● Do not use with anything other than the accessories supplied with this product. 	
 Required Action	<ul style="list-style-type: none"> ● Before plugging the power cord into the wall outlet, make sure that the main POWER switch is set to OFF. <ul style="list-style-type: none"> → May result in damage to the chair or electric shock. ● Make sure that the plugs on the power cord are inserted fully into the cord connector and the wall outlet. <ul style="list-style-type: none"> → May result in electric shock, short circuit, or combustion. ● Before using the chair for the first time, turn on all the switches in order as indicated in this Operating Manual to check that the chair is operating properly. ● Start off with a lighter massage. <ul style="list-style-type: none"> → Starting with a deep massage may result in injury. Elderly people and persons with weak bones should be especially careful when selecting massage intensity. ● For safety, first check the position of the mechanism. If the mechanism is not in the storage position, turn the main POWER switch ON and check that the mechanism returns to its storage position before sitting down slowly. For the storage positions, see page 7. ● Do not climb onto, stand on, sit on or otherwise subject the leg rests to a heavy load. <ul style="list-style-type: none"> → Personal injury or damage to the chair may result if the chair tips over. ● If you use the chair after it has not been used for a long time, read this Operating Manual again thoroughly and check if the chair operates properly. 	

Safety Precautions: During Use

Warning

 Prohibited	<ul style="list-style-type: none"> ● Do not wrap the power cord or the remote control cord around the chair, or place the chair on top of either cord. Do not scratch, tear, treat, unduly twist, stretch or bend the power cord or the remote control cord. <ul style="list-style-type: none"> → The cords may be damaged, resulting in fire or electric shock. ● Do not spill water and/or other liquids on the chair or remote control. <ul style="list-style-type: none"> → May result in electric shock, short circuit, or combustion. ● Children and those who are unable to express their intentions should not use the chair. Keep children and pets off the chair and away from the surrounding area (especially behind the reclining seat back or under the leg rest). <ul style="list-style-type: none"> → Could result in accident or injury. ● Do not use the chair while more than one person is sitting in it. In particular, do not use the chair while holding a small child (or baby). <ul style="list-style-type: none"> → Could result in accident or injury. ● Do not use the chair while your body is wet, or with wet hands. <ul style="list-style-type: none"> → Could result in electric shock or damage to the chair. ● Do not use the chair if the fabric is torn. If it is, unplug the power plug from the wall outlet. <ul style="list-style-type: none"> → Could result in an accident, injury or electric shock. Contact your local distributor for repairs. ● Do not release your hands suddenly while lifting the chair by hand. 	
 Required Action	<ul style="list-style-type: none"> ● Make sure no one, especially children or pets, is around the chair before starting a massage program or when lowering or raising the reclining seat back. Be careful that people or objects are not caught behind the seat back or under the leg rest. <ul style="list-style-type: none"> → Could result in accident or injury. ● If a person or object is trapped, press the recline button to return the chair to a safe position. 	

SAFETY PRECAUTIONS

SAFETY PRECAUTIONS

Safety Precautions: During Use

⚠ Caution

 <p>Prohibited</p>	<ul style="list-style-type: none"> ● Do not use the chair unclothed. ● Do not use the chair for purposes other than massage, or other than the ordinary uses of a chair. → Could result in accident or injury. ● Do not use the chair to massage the head, chest, stomach, and/or joints (knees, elbows). → May result in nausea or injury. Place a soft towel behind your neck if you want to avoid intense stimulation of the area. ● Do not place your hands into the roller tracks, or the moving parts of the reclining seat back or other areas. Do not sit wearing hard objects, such as a hair accessory, necklace, watch, bracelet or ring. → May result in personal injury. ● Do not place your legs anywhere other than the active area (leg mechanism) described in this Operating Manual. ● Do not place anything other than legs and feet on the leg rest. → May result in personal injury. ● Do not operate the massage chair or leave it operating when there is no one in it. → Could result in accident or injury.
 <p>Required Action</p>	<ul style="list-style-type: none"> ● In an emergency, or if you find any abnormal condition, push the EMERGENCY STOP button on the remote control to stop operation immediately. ● If you get acute pain or other unusual physical discomfort while using the chair, immediately discontinue the use of the chair and seek medical advice. ● If you feel the massage is too strong, reduce the stimulation according to this Operating Manual. If this solution does not work, stop using and contact your local distributor. → May result in injury if you continue use with high stimulation under such circumstances. ● If you feel that the massage is too intense after removing the buffer pad, stop use immediately and reinstall the buffer pad. → May result in personal injury. ● Do not operate a massage without the back pad. ● Do not fall asleep in the chair during a massage session. → May result in personal injury. ● Keep massage sessions to no longer than 30 minutes. Do not exceed 5 minutes for an area-specific massage of any particular part of the body. → Continuous use for extended periods of time can result in over-stimulation, which could cause the opposite of the desired effect, or even injury. Limit the massage time to a total of about 30 minutes in a day. ● If your leg or foot comes out of the leg rest during the massage, do not try to force it back in. → Undue force may result in damage to the mechanism.
 <p>Unplug the chair</p>	<ul style="list-style-type: none"> ● If there is a power outage or the possibility of a power outage, stop operation immediately and turn the main POWER switch to OFF. Unplug the power plug from the wall outlet. → Accident or injury may result if the chair is still in operating mode when the power resumes. ● If there is a threat of lightning, stop operation immediately and turn the main POWER switch to OFF. Unplug the power plug from the wall outlet. ● If an operation does not start, or if you find any abnormality, stop the chair immediately and turn the main POWER switch to OFF to prevent an accident. Unplug the power plug from the wall outlet. Contact your local distributor for inspection or repairs. → Electric shock or fire caused by a short circuit may result if the chair is left plugged in under such circumstances.

Safety Precautions: After Use or When Not in Use

⚠ Caution

 <p>Prohibited</p>	<ul style="list-style-type: none"> ● Be sure to turn the main POWER switch to OFF after every use and to unplug the power plug from the wall outlet. → An accident or injury may result if children or pets play with the chair. ● Do not use a power cord for anything other than the chair. ● After using the chair, be sure to fully return the reclining seat back to the upright position. ● Wipe the chair off after every use, and store the chair in a place with low humidity. ● Do not place any object on the chair.
 <p>Unplug the chair</p>	<ul style="list-style-type: none"> ● Hold the power plug when unplugging from the wall outlet. Do not pull on the power cord. → May result in electric shock or fire caused by short-circuiting. ● Before cleaning the chair, make sure to unplug the power plug from the wall outlet. Do not plug or unplug the power plug with wet hands. ● If the chair is left unused for a while, be sure to unplug it from the wall outlet. → Insulation degradation by dirt or moisture may cause electric shock, or fire due to short-circuit.

Other Precautions

⚠ Warning

 <p>Prohibited</p>	<ul style="list-style-type: none"> ● Do not stand on the chair. Do not make a major impact or sit on any part of the chair other than the seat. → Personal injury or damage to the chair may result if the chair tips over.
 <p>Do not disassemble</p>	<ul style="list-style-type: none"> ● Do not try to repair any malfunction of the chair. Contact your local distributor for assistance. ● A damaged power cord is dangerous. If the power cord is damaged, contact your local distributor for replacement. ● Do not modify or remodel the chair. → May result in fire, electric shock, or injury.

⚠ Caution

 <p>Required Action</p>	<ul style="list-style-type: none"> ● If you lend, sell, or otherwise transfer the chair to another party, make sure to include this Operating Manual.
--	---

Note

- Before disposing of the product or its accessories, consult your local authorities about the proper procedure.

MAIN FEATURES & BENEFITS

Main Features

New, full-body approach to massage

Full body massage with a double active area mechanism

Two active area mechanisms are used, for the back and legs, and their operations can be synchronized for integrated stretching. A centripetal method that returns the blood flow from the extremities of the body back towards the heart and a centrifugal method that induces the blood to flow from the heart to the extremities of the body are also available.

In addition, focused massages can be performed for your upper body and lower body at the same time, giving you a thorough massage.

Legs held tight for kneading

Ring shiatsu mechanism for the legs

A ring-shaped shiatsu mechanism that we developed with our own technology holds the calves and other leg areas firmly for kneading.

Finding shiatsu points over the entire body

An automatic full body shiatsu point search system using optic sensors

Using optic sensors that we developed with our own technology, the shiatsu points that vary from person to person are found before starting the massage.

Even asymmetrical knotting can be relieved

An active area with independent left and right push-out for kneading and tapping

The independent left and right air push-out mechanisms can perform a focused massage of asymmetrical knotting or fatigue in the upper body.

A posture that feels weightless

Zero gravity angle

The reclining seat back angle during a massage is set automatically to an angle that minimizes the load on muscles.

The massage is performed in a comfortable posture that almost feels like weightlessness.

Firmly held for shiatsu

Sole massage

The ends of your feet are held for a massage that stimulates your soles using the ring-shaped shiatsu mechanism.

Relieve any kind of fatigue

Wide range of programs

The preset programs allow you to select the optimal program according to your purpose, while the manual programs allow you to freely combine different operations.

Fingers

Heater function

Heats your extremities to relax your body.

Benefits


As a therapeutic massage device in the general households, the INADA chair provides the following benefits.

- Relaxes muscles
- Relieves minor muscle aches and pains

COMPONENTS & DESCRIPTION

Chair

COMPONENTS & DESCRIPTION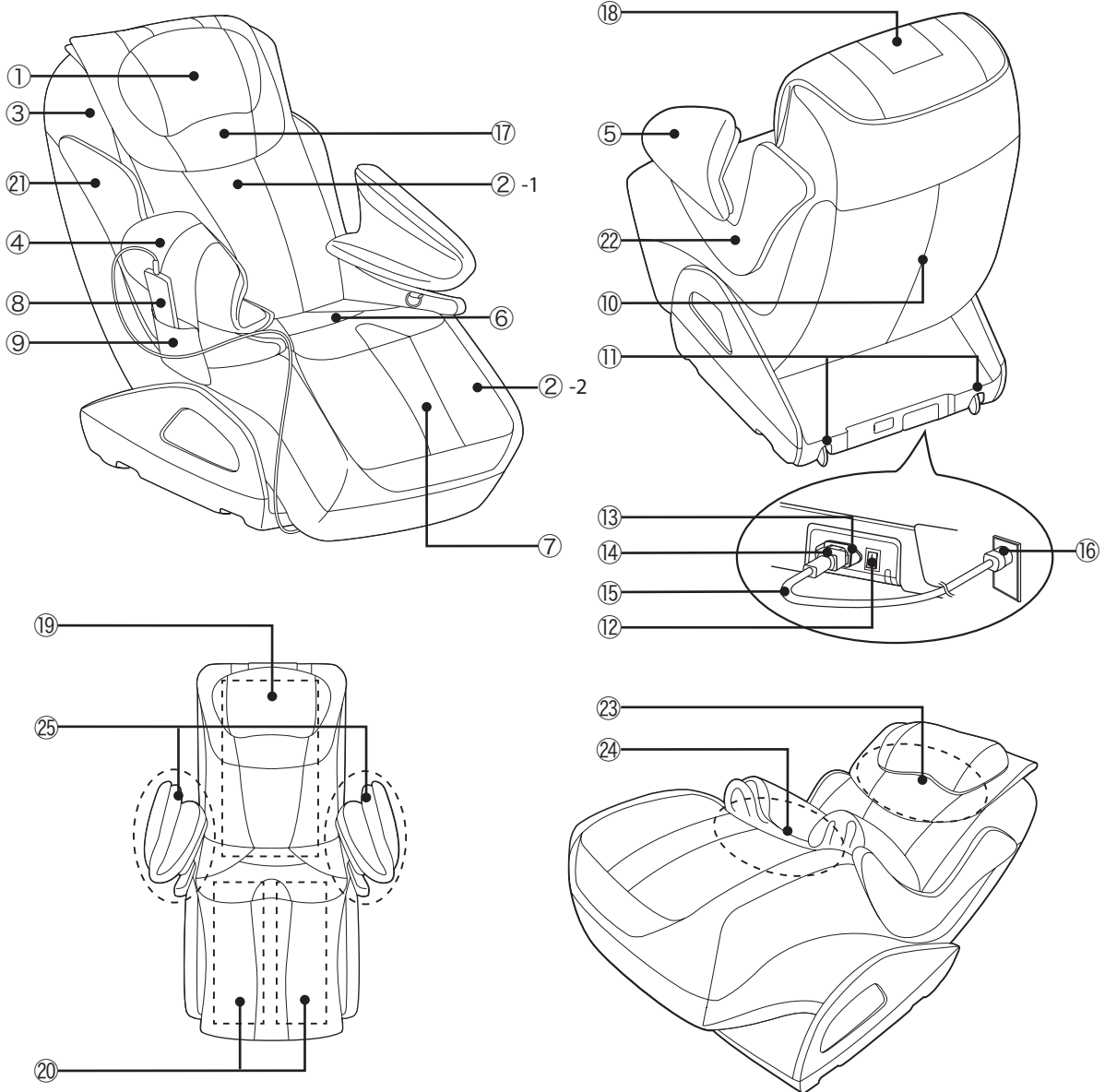


①	Headrest (with headrest cover)	
②	Pads	
	② -1	Back Pad
	② -2	Leg Pad
③	Reclining Seat Back	
④	Armrest (right)	
⑤	Armrest (left)	
⑥	Seat	
⑦	Leg Rest	
⑧	Remote Control	

⑨	Remote Control Pocket
⑩	Back Cover
⑪	Wheels
⑫	Main POWER Switch
⑬	Power Connector
⑭	Connector
⑮	Power Cord
⑯	Plug
⑰	Headrest (large)
⑱	Headrest Cover

⑲	Active Area (back mechanism)
⑳	Active Area (leg mechanism)
㉑	Armrest Anchor (right)
㉒	Armrest Anchor (left)
㉓	Storage Position (back mechanism)
㉔	Storage Position (leg mechanism)
㉕	Air Cushion


Remote Control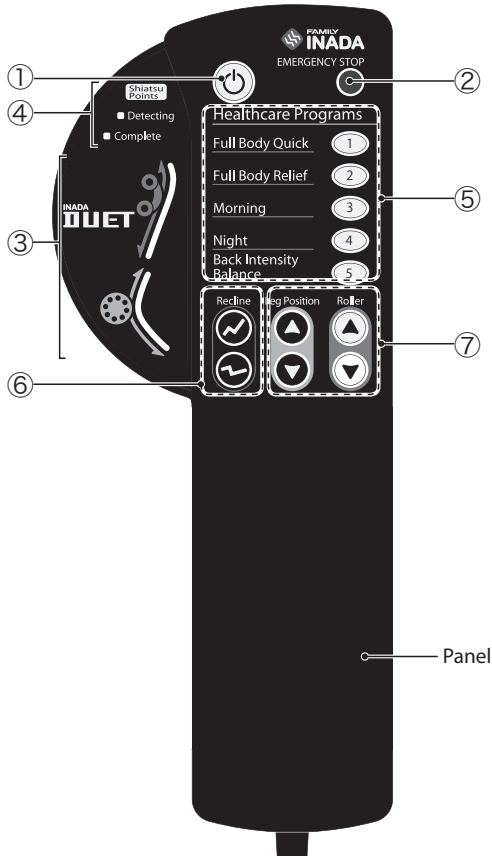

①	POWER Button	Turns the remote control power ON and OFF. Pushing the POWER button while the chair is activated causes the rollers to return to the storage position, after which the chair turns itself off. The button lights in green while the power to the remote control is ON, lights in orange while the power is OFF, and flashes in red while the chair is stopped in an emergency.
②	EMERGENCY STOP Button	Use this button to shut the chair off immediately in an emergency or if there is a malfunction during operation of the chair. The rollers will stop immediately.
③	Roller Position Display	The position of the roller during the massage is displayed.
④	Shiatsu Points Locator Indicator	A light indicates when the shiatsu point search is operating and when the search has finished.
⑤	Preset Program Buttons 1	Use to select the corresponding program.
⑥	Recline Button	Use to adjust the angle of the reclining seat back.
⑦	Position Adjustment Buttons (Shoulders & Legs)	Use to adjust the shoulder position while the shiatsu point locator is operating, and to adjust the roller position during manual selection mode.

COMPONENTS & DESCRIPTION

Remote Control/Inside the Panel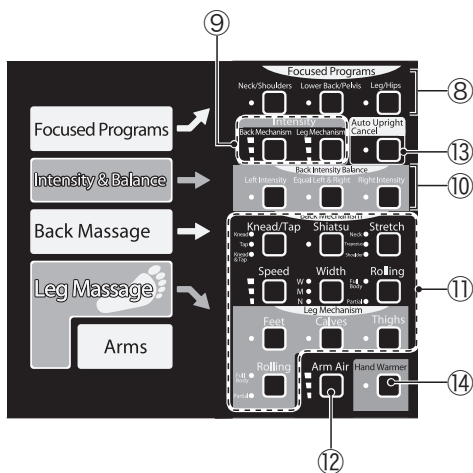

⑧	Preset Program Buttons 2	Use to select the corresponding program.
⑨	Intensity Adjustment Buttons	Use to adjust the intensity of the back mechanism and leg mechanism.
⑩	Balance Adjustment Buttons	Use to set the balance between the left and right massage of the back mechanism.
⑪	Manual Selection Buttons	Use to select basic operations or special operations.
⑫	Arm Air Massage Button	Use to select an air massage of the arms.
⑬	Auto Upright Cancel Button	The automatic seat reclining raise operation when the course is completed can be set ON/OFF.
⑭	Hand Warmer Button	Use to switch the hand warmer ON/OFF.

Warning	<ul style="list-style-type: none"> Do not spill water and/or other liquids on the remote control. Do not handle the remote control with wet hands.
Caution	<ul style="list-style-type: none"> Do not sit on the remote control. Do not drop or step on the remote control. Do not pull the remote control cord forcibly. Do not use pointed or hard objects to push the buttons. <p>→ May result in damage to the remote control.</p>

BEFORE USE

- This product is shipped with the reclining seat back in its fully reclined position. Be sure to raise the seat back and armrests.
- Do not expose the chair to sudden bursts of heat. If the room where the chair is to be used is cold, turn the heat up slowly. If the chair is moved from a cold place into a heated room, wait one hour before using the chair.
→ Condensation could occur in the moving parts of the chair, which may compromise performance.

Installation


Allow for at least 40 cm (16 inch) clearance behind the chair and enough space in front of the chair for people to pass by, to enable the seat back to be fully reclined without obstruction.

BEFORE USE

Caution


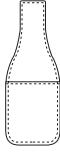
- Do not install the chair in an area of high humidity, such as a bathroom or sauna.
- Install on a flat, level surface only.
- Do not install the chair where it can come into contact with direct heat, such as direct sunlight or a radiator.
- Keep fire sources such as cigarettes and ashtrays away from the chair.

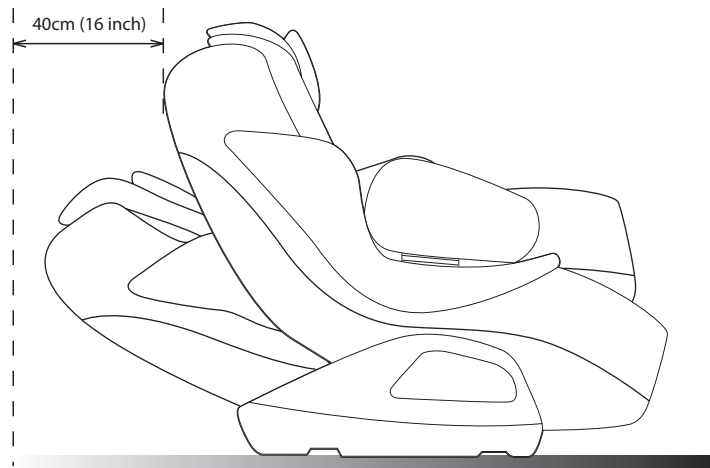- To avoid damaging floor surfaces and to minimize the chair noise, it may be better to install the chair on a carpeted surface or to lay a piece of carpet under the chair.
(Do not use electric blanket.)
- The chair is very heavy, so please be careful when unloading it so as not to damage floor surfaces.
- Install the chair at least 1 m (39 inch) away from TVs, radios, and other audio or video equipment. Otherwise they may make noises affected by the operation of the chair.
- The load to the floor per 1 cm² (0.4 inch²) is approximately 1.0 kg (2.2 lb) with a person weighing 100 kg (220 lb) sitting on the chair.



Checking the Supplied Items

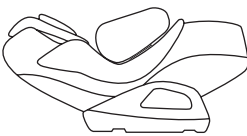Check that the chair and the other items are all in the package.

Chair	Items Included			
	 Operating Manual	 Buffer Pad	 Power Cord	 Remote Control Pocket

How to Transport or Move the Chair

Turn the main POWER switch to OFF and disconnect the plug from the wall outlet before moving the chair.

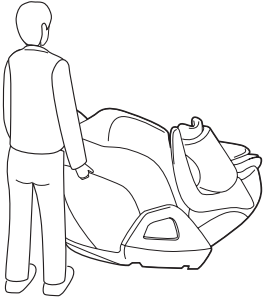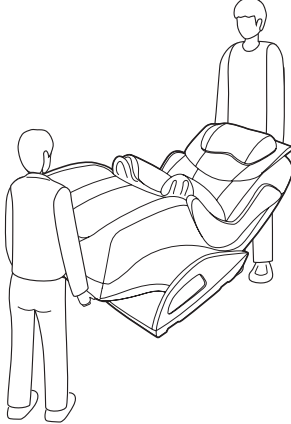To Move with the Wheels	To Carry
Fully recline the seat back, hold the handle on the rear of the leg rest, and then move the chair on the wheels by lifting and holding up the front part of the chair.	Fully recline the seat back and carry the chair by holding the handle on the upper part of the back cover and the rear of the leg rest.
	

BEFORE USE

- The chair is very heavy, so be careful not to hurt your lower back when transporting or moving it.
- The remote control is attached to the chair, so make sure that it does not fall when moving the chair.


Caution

- Do not move the chair while someone is in it.
- Be sure to disconnect the power cord from the wall outlet before moving the chair.
→ The power cord may be damaged, resulting in fire or electric shock.


How to Raise the Armrests

Raise the armrests with the following procedure.

① Raise the folded armrests.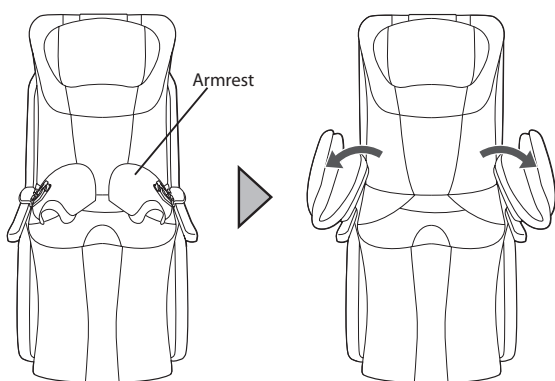

② Rotate the stopper to apply the lock.
(For safety, rotate as far as it will go.)

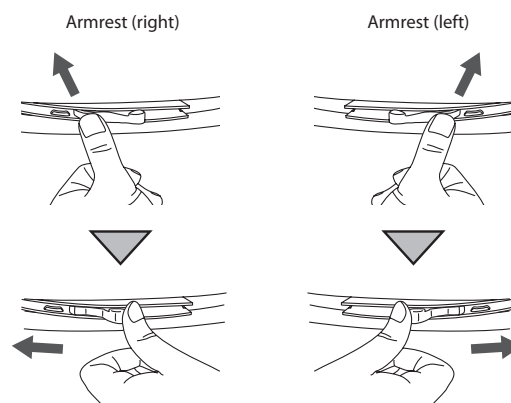How to Fold the Armrests

Fold by following the raising procedure in reverse.

Caution

- Be careful not to trap your hands or fingers when raising or folding the armrests.
- If you use the chair while the armrests are not locked, they may fold accidentally.
→ May result in accident or injury.


BEFORE USE

How to Install the Remote Control Pocket


The remote control pocket can be installed on an armrest.

While the armrest is raised, firmly install the remote control pocket onto the velcro on the rear side of the armrest. To remove, perform the installation procedure in reverse.

BEFORE USE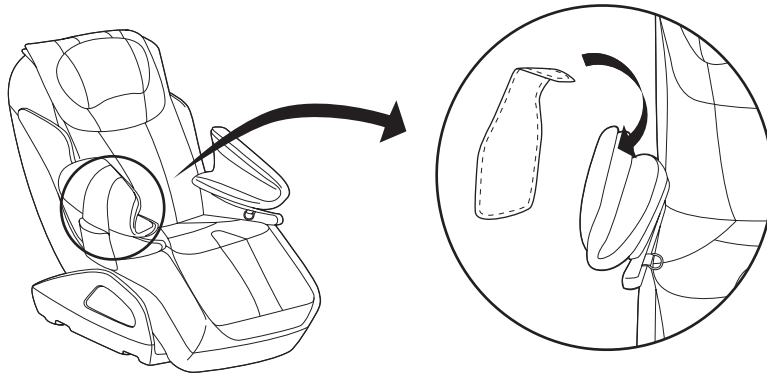

Use with the remote control inserted into the remote control pocket as shown in the figure on the right.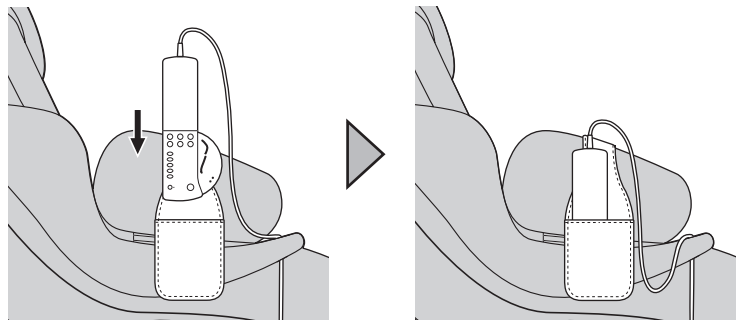


⚠ Caution


● When the remote control is inserted into the remote control pocket, be careful of the remote control cable.

How to Connect the Main Power

① First, plug the power cord into the cord connector located at the back of the chair. Then, plug the other end into the wall outlet.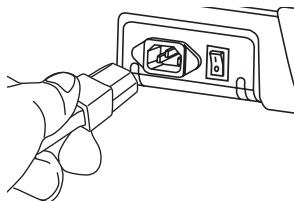

② Set the main POWER switch located at the back of the chair to ON.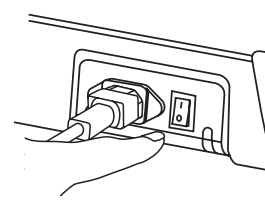

*If the main POWER switch is turned ON while the rollers are stopped partway, they will automatically return to the storage position and then stop.

⚠ Warning

● Do not use with any type of transformer device.
→ May result in damage to the chair or electric shock.

⚠ Caution

● Before plugging the power cord into the wall outlet, make sure that the main POWER switch is set to OFF.
● Make sure that the plug on the power cord is inserted fully into the wall outlet.
● Before using the chair for the first time, turn on all the switches in order as indicated in this Operating Manual to check that the chair is operating properly.


Checking the Chair Fabric

Check the fabric in all areas for tearing.

① Back Pad


Open the top zipper.

Fold the back pad forward, and check with your hand that the fabric is not damaged.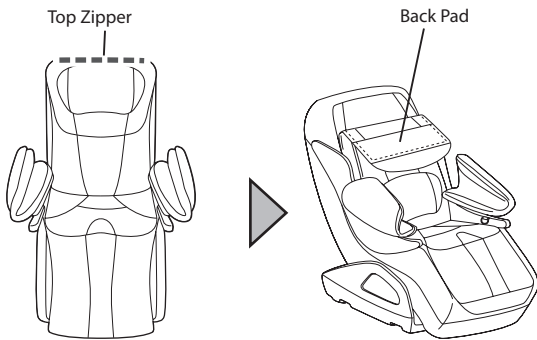

② Leg Pad

Check with your hand that the fabric is not damaged from the open areas on the sides of the leg pad.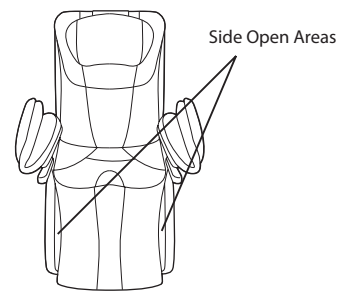



BEFORE USE

⚠ Caution

● Before each use, check for holes or tears in the fabric of the active area. Also, check the other parts for tearing. Unplug the chair and contact your local distributor for assistance. Never use the chair with the fabric of the active area removed or with holes or tears in the fabric, because doing so is extremely dangerous.

Laying Out the Remote Control Cord (Same for Left and Right)

① Pass the remote control cord through the Velcro on the rear of the foot area and secure it.

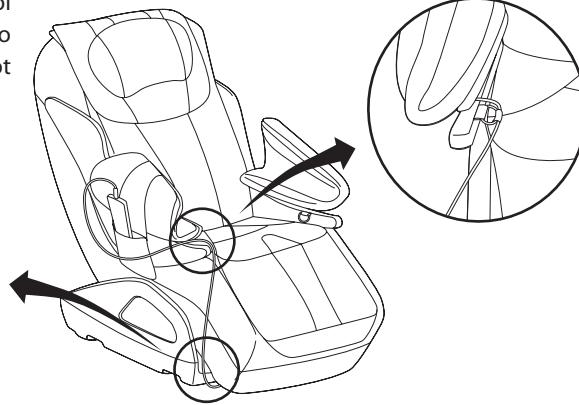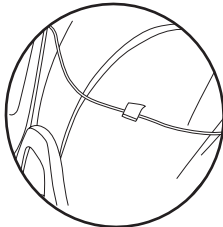② Pass the velcro attached to the remote control cord through the ring at the end of the armrest anchor and secure it.

③ Insert the remote control into the remote control pocket.

POSITION ADJUSTMENT

How to Sit

Normally, the mechanism is drawn back into the storage position. For safety, first check the position of the mechanism. If the mechanism is in any position other than the storage position, turn the main POWER switch ON before sitting down to allow the mechanism to return to its storage position. When you finish checking, sit down slowly in the center of the seat.


Caution

- **Do not sit down on the chair while the seat back is reclined.**
→ Any sudden impact on the reclining seat back or leg rest could cause damage to the mechanism. Also, an accident or injury may result if the chair tips over.
- **Start off with a lighter massage.**

Precautions for Getting On or Off the Chair

When getting on or off the chair, do not support yourself by placing your hands onto the air cushions attached to the armrests. May result in a mechanical failure. Support yourself when getting on or off the chair by placing your hands on the end of the armrest anchor.

POSITION ADJUSTMENT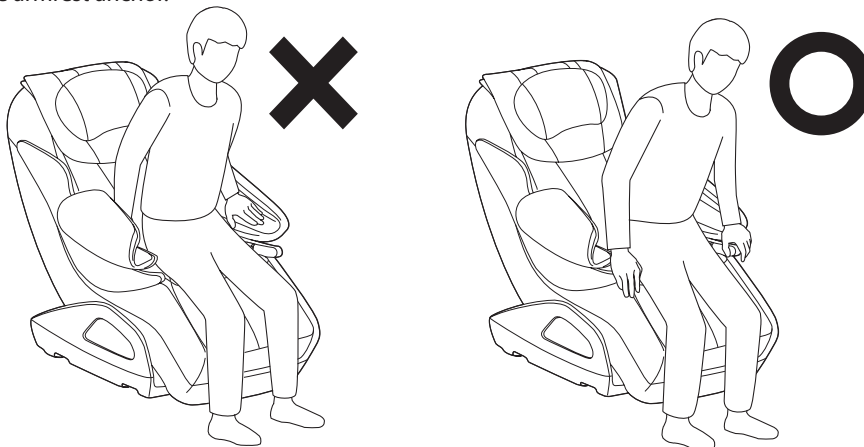


Warning

- **Do not subject the armrests to strong impacts from above.**
→ Personal injury or damage to the chair may result if the chair tips over.

Headrest Adjustment

The headrest is attached to the chair with Velcro on its rear side so that you can adjust the position. Adjust the headrest to match the position of your head before starting the massage.

Adjust the headrest position by changing the position it is attached to the velcro on the chair.

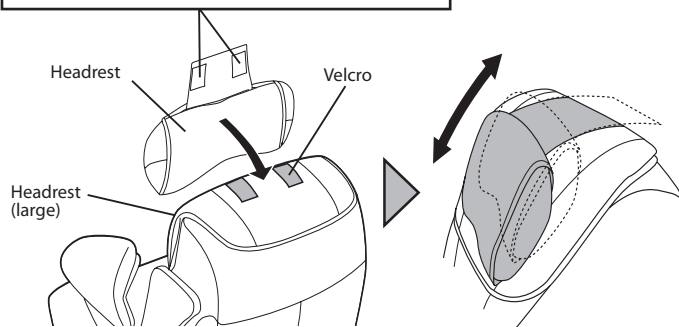Proper Position of Headrest

Do not place at the rear of your neck.


Place it at the rear of your head (occipital).

- **Place the headrest in the proper position. If it is not placed properly, the automatic shiatsu point locator may not function properly.**


How to Remove the Headrest Cover

The headrest cover attached to the headrest can be removed with the following procedure.

① Remove the headrest from the chair.

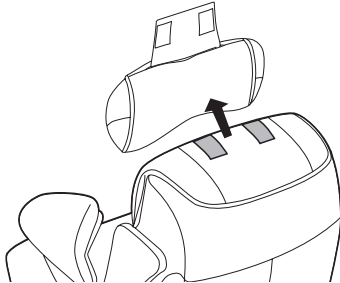
② Remove the band-shaped part of the headrest from the open area in the center of the headrest cover, and pull out the headrest from the open area on the side of the headrest cover.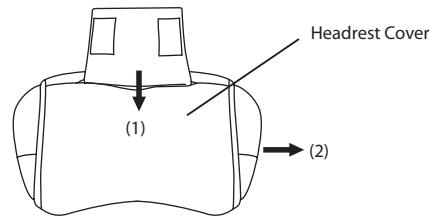

How to Use the Buffer Pad

If you feel that the massage is too intense, place the supplied buffer pad onto the back pad or seat.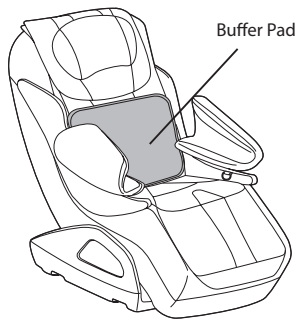

POSITION ADJUSTMENT

⚠ Caution


- Make sure that the buffer pad does not curl up or become lop-sided.
→ May result in accident or injury.

How to Use the Armrest

Place your arm as shown in the diagram below so that your elbow is between the armrest and reclining seat back.

Place your elbows between the armrest and reclining seat back.

Insert so that your palm is facing down.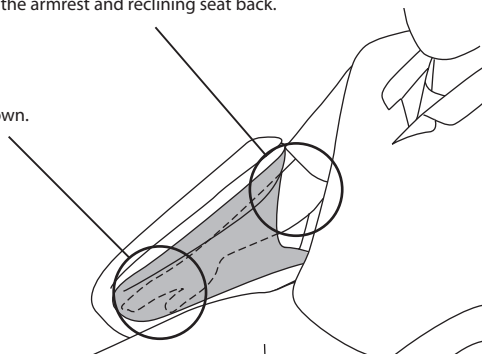

⚠ Caution

- Do not use for anything other than your arms.
→ May result in personal injury.
- If your arm comes out of the armrest during the massage, do not try to force it back in.
→ Undue force may result in damage to the mechanism.
- Do not massage your elbows. Also, be sure that your palms are facing down when massaging your arms. Do not insert your arm in the other direction.
→ May result in personal injury.

POSITION ADJUSTMENT

How to Use the Reclining Seat Back

POSITION ADJUSTMENT


Vertical Adjustment

- To raise the reclining seat back

Press the top recline button to raise the reclining seat back.

- To lower the reclining seat back

Press the bottom recline button to lower the reclining seat back.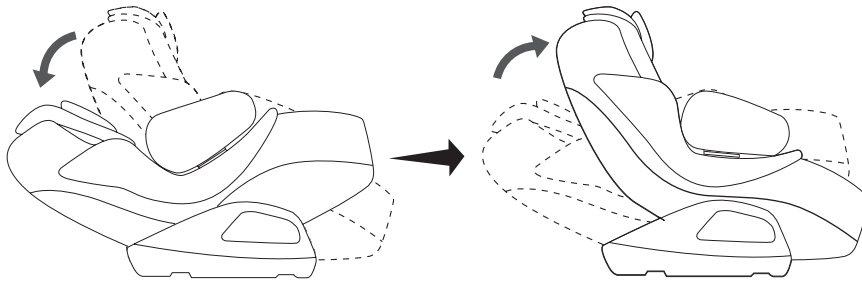

AFTER USE

Hold Function

Press a recline button for 2 seconds or longer to move the reclining seat back automatically to its maximum position.

While the reclining seat back is moving, you may feel a slight rocking.


This occurs when the center of gravity shifts while the reclining seat back is moving.

Warning


- Make sure no one, especially children or pets, is around the chair before starting a massage program or when lowering the reclining seat back. Be careful that people or objects are not caught behind the seat back or under the leg rest.

How to Use the Leg Rests in the Soles Program

When using the soles program, check that the leg mechanism is in position, and then place your feet between the rollers as shown in the figure below.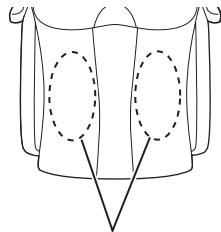

Insert your heels here.

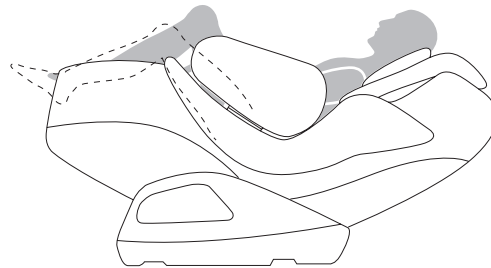AFTER USE

While making sure there is nothing in the surrounding area, fully return the reclining seat back to its upright position.

Set the main POWER switch located at the back of the chair to OFF.

Unplug the power plug from the wall outlet. This will save energy because no standby power will be consumed.

Caution

- Be sure to turn the main POWER switch to OFF after every use and to unplug the power plug from the wall outlet.
→ An accident or injury may result if children or pets play with the chair.
- After using the chair, be sure to fully return the reclining seat back to the upright position.
- Wipe the chair off after every use, and store the chair in a place with low humidity.
- Hold the power plug when unplugging from the wall outlet. Do not pull on the power cord.
- If the chair is left unused for a while, be sure to unplug it from the wall outlet.
- Be careful not to tangle yourself in the cords and trip over.

PROGRAMS & SELECTION

Preset Programs

The preset programs make it easy to select a program that suits your preferences or objectives.

Healthcare Programs

Full Body Quick

This program is recommended for recovering from muscle fatigue over the entire body in a short time.

Full Body Relief

The purpose of this program is to recover from muscle fatigue over the entire body. The program performs integrated stretching for the shiatsu points over the entire body, from the base of the neck to the ankles.

Morning

Massages along the veins, using a centripetal method that returns the blood flow from the extremities of the body back towards the heart. This program is recommended for refreshing or awakening the body.

Night

Massages along the arteries, using a centrifugal method that induces the blood to flow from the heart to the extremities of the body. This program is recommended before sleeping to relax the body.

Back Intensity Balance

This program adjusts the left and right balance of the body by automatically adjusting the left and right roller push-out to perform a massage that focuses individually on the left and right of the back.

Focused Programs

Neck / Shoulders

This massage focuses on the base of the neck, back of the neck, top of the shoulders, and shoulders.

Lower Back / Pelvis

This massage focuses on the area from the lower back to the buttocks, and on the thighs.

Legs / Hips

This massage focuses on the area from the calves to the thighs, and on the buttock area.


Manual Selection

You can combine your favorite massage types. You can also adjust the speed, intensity, and roller positions.

PRESET PROGRAMS

Preset Program Messages

- 8 automatic programs provide a variety of different massages to relieve fatigue.
- With the preset program buttons, you can choose your favorite program to give you a massage according to the preset menu.

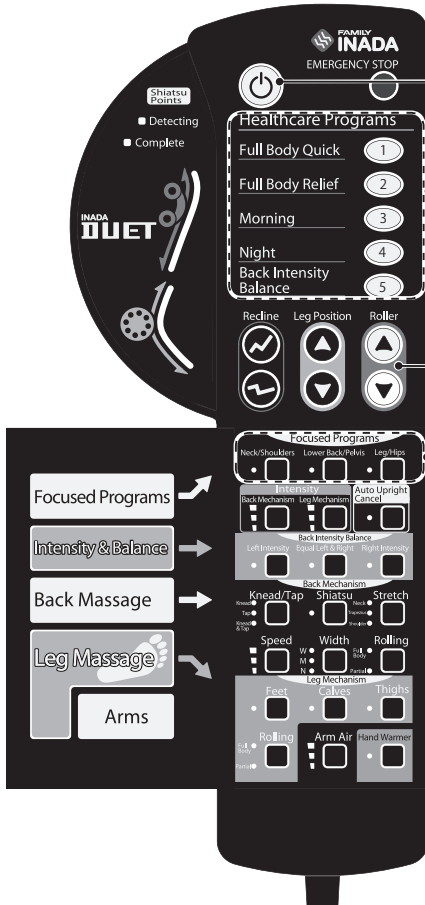1 Press the POWER button.

If no selection is made within 5 minutes, the power shuts off automatically.

2 Press the desired program button.

The reclining seat back moves until it reaches the zero gravity angle*. Then, the shiatsu point locator indicator turns on and shiatsu point locating is performed.

[Note]

Sit as far back as possible in the chair and put your head against the headrest in a relaxed manner. Shiatsu points cannot be accurately located if you are not properly seated.



3 The shoulder positions can be adjusted.

After the shiatsu points are located, the roller buttons flash.


The shoulder positions can be adjusted while the buttons flash. If no adjustment is required, do not use these buttons.

Shoulder Position Adjustment

If the shoulder positions are not correct, adjust the shoulder positions while the roller buttons are flashing.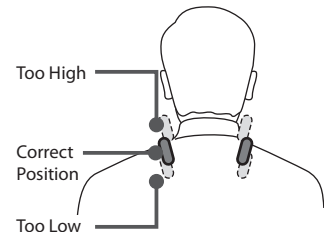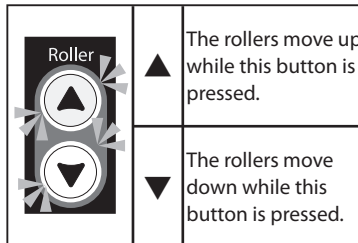


* About the Zero Gravity Angle

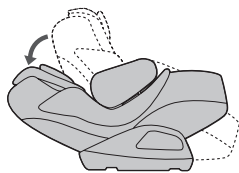This is a reclining angle that minimizes the load on the muscles, using a posture that brings to mind a feeling of weightlessness.

4 The program starts.

When shiatsu point locating is completed, the Search Complete light turns on and the program starts. During program operation, you can adjust the mechanism intensity, adjust the air intensity, or cancel the air operation.

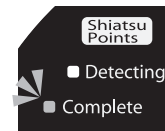Mechanism Intensity Adjustment

The mechanism intensity can be adjusted in 3 levels.

Each time the button is pressed, the intensity changes in the order of Low, Medium and High. (The default setting when the program starts is Medium.)

Air Massage Intensity Adjustment and Cancellation

The air massage intensity can be adjusted. Each time the air intensity button is pressed, the intensity changes in the order of Low, Medium and High. (The default setting when the program starts is Medium.)

Back Intensity Balance Function

Left Intensity	Left intensity reduces the number of massages on the right side of the back, and increases the number of massages on the left side of the back.
Equal Left & Right	This setting keeps the same balance for the massages on the left and right of the back.
Right Intensity	Right intensity reduces the number of massages on the left side of the back, and increases the number of massages on the right side of the back.

5 The program finishes.

The reclining seat back automatically rises, and the rollers return to their storage position. Then, the power to the remote control is shut off. However, the reclining seat back can still be adjusted up or down.

Auto Upright Cancel

The automatic rising of the reclining seat back after the program is completed can be canceled. A green lamp lights while this function is canceled. Press the button again to release the cancellation and raise the reclining seat back when the program is completed.

Changing Programs

Before the program in progress finishes, you can change it to another program. After a total duration of 30 minutes, the course in progress stops even if not completed.

Switching from One Preset Program in Progress to Another

When the button of one preset program is pressed while another preset program is in progress, the rollers return to the detected shoulder positions, and the shoulder positions are checked again before starting the new program. The shoulder positions can be adjusted while the shoulder positions are being checked again.

Switching from a Preset Program to a Manual Selection

When one of the buttons for a manual selection located inside the panel of the remote control is pressed while a preset program is in progress, the preset program immediately stops, and the selected operation starts. If an operation with shoulder detection is selected, the operation starts after checking the shoulder positions again. (To select an air massage, first select an operation such as kneading or tapping, or press the **POWER** button to stop the preset program in progress before making the selection.)

Stopping in the Middle of a Massage

To stop mid-massage, press the **POWER** button. The rollers return to their storage positions and stop.

In an emergency or if there is some malfunction during the operation of the chair, press the **EMERGENCY STOP** button. The rollers will stop immediately. After confirming safety, press the **POWER** button. The rollers return to their storage positions and stop.

When a preset program is selected, the reclining seat back may recline automatically. Also, the reclining seat back moves automatically during some programs. For this reason, before using the chair, make sure that there is no person or obstruction behind it. Also, do not let other people approach the chair during use.

Caution

- If you feel the massage is too strong, reduce the stimulation according to this Operating Manual. If this solution does not work, stop using and contact your local distributor.
→ May result in injury if you continue use with high stimulation under such circumstances.
- Do not place your hands or feet into the roller tracks, or the moving parts of the reclining seat back or other areas. Do not sit wearing hard objects, such as a hair accessory, necklace, watch, bracelet, or ring.
- If your leg or foot comes out of the leg rest during the massage, do not try to force it back in.
- Keep massage sessions to no more than 30 minutes in length. Do not exceed 5 minutes for an area-specific massage of any particular part of the body.
→ Continuous use for extended periods of time can result in over-stimulation, which could cause the opposite of the desired effect, or even injury. Limit the massage time to a total of about 30 minutes in a day.

MANUAL SELECTION

Operation of Parts and Position Adjustment

1 Press the POWER button.
If no selection is made within 5 minutes, the power shuts off automatically.

2 Press the desired button.
Select a combination of your operations of choice from basic operations, rolling, special operations and air massage.

3 The shoulder positions can be adjusted.
When an operation is selected that includes shiatsu point locating, the roller buttons flash after the shiatsu points are located. The shoulder positions can be adjusted while the buttons flash. If no adjustment is required, do not use these buttons.

4 The program starts.
When an operation is selected that includes shiatsu locating, the Search Complete light turns on and the program starts after the shiatsu point locating is completed. A basic operation can be changed, the mechanism intensity can be adjusted, and the air can be canceled or its intensity adjusted even while a program is operating.

5 The program finishes.
The rollers return to their storage positions. Then, the power to the remote control is shut off. However, the reclining seat back can still be adjusted up or down.

Roller Position Adjustment		
	When rolling is not selected: The roller position can be adjusted to the area that you want to massage.	<p>▲ The rollers move up while this button is pressed.</p> <p>▼ The rollers move down while this button is pressed.</p>
	When full body rolling is selected: Changes between up and down.	<p>▲ When pressed during a down movement, changes to up.</p> <p>▼ When pressed during an up movement, changes to down.</p>
	When partial rolling is selected: The center position of the range of partial rolling can be adjusted. The position at which you release the button becomes the center of the operation.	<p>▲ The rollers move up while this button is pressed.</p> <p>▼ The rollers move down while this button is pressed.</p>

● When up (or down) is selected when the roller is already at the highest (or lowest) position or in the shoulder position where the shiatsu points were located, a warning alarm sounds.

Width Adjustment

During a basic operation where width adjustment is possible, the width of the rollers can be adjusted in 3 steps (narrow, medium and wide).
(The default setting when the program starts is narrow.)

Speed Adjustment

During a basic operation where speed adjustment is possible, the speed of action can be adjusted in 3 steps.
(The default setting when the program starts is the medium level.)

Mechanism Intensity Adjustment

The mechanism intensity can be adjusted in 3 levels.
Each time the button is pressed, the intensity changes in the order of Low, Medium and High.
(The default setting when the program starts is Medium.)

Arm Air Massage Intensity Adjustment

The arm air massage intensity can be adjusted.
Each time the air intensity adjustment button is pressed, the intensity changes in the order of Low, Medium and High.
(The default setting when the program starts is Low.)

Changing Programs

Switching from a Manual Selection to a Preset Program

If a preset program button is pressed while a manual selection is in progress, the selection switches to the preset program. After a total duration of 30 minutes, the program in progress stops even if not completed.

Stopping in the Middle of a Massage

To stop mid-massage, press the **POWER** button. The rollers return to their storage positions and stop.

In an emergency or if there is some malfunction during the operation of the chair, press the **EMERGENCY STOP** button. The rollers will stop immediately. After confirming safety, press the **POWER** button. The rollers return to their storage positions and stop.

Caution

- **If you feel the massage is too strong, reduce the stimulation according to this Operating Manual. If this solution does not work, stop using and contact your local distributor.**
→ May result in injury if you continue use with high stimulation under such circumstances.
 - **Do not place your hands or feet into the roller tracks, or the moving parts of the reclining seat back or other areas. Do not sit wearing hard objects, such as a hair accessory, necklace, watch, bracelet, or ring.**
 - **If your leg or foot comes out of the leg rest during the massage, do not try to force it back in.**
 - **Keep massage sessions to no more than 30 minutes in length. Do not exceed 5 minutes for an area-specific massage of any particular part of the body.**
- Continuous use for extended periods of time can result in over-stimulation, which could cause the opposite of the desired effect, or even injury. Limit the massage time to a total of about 30 minutes in a day.

MANUAL SELECTION

Description of Manual Program Operations

You can select massaging actions of your choice from the manual selection massage options.

- A circle in the adjustment column means that the particular adjustment can be made in the course of that massage action.
- The width cannot be adjusted if an action for which width adjustment is not available is combined with rolling.

Back Mechanism

	Function	Action	Speed Adjustment	Width Adjustment	Up/Down Adjustment	Intensity Adjustment
Basic Operations	Kneading	Performs upward kneading and downward kneading alternately.	○	—	○	○
	Tapping	This performs a tapping type of massage.	○	○	○	○
	Kneading & Tapping	Performs upward kneading and tapping simultaneously.	○	—	○	○
Rolling	Full Body Rolling	Performs stretching of the upper body using up and down movements.	—	○	○	○
	Partial Rolling	Performs partial stretching of the upper body using up and down movements.	—	○	○	○
Special Operations	Shiatsu	Performs Shiatsu massage.	—	—	○	○
	Neck Stretch	Performs stretching centered on the neck.	—	—	—	○
	Trapezius Stretch	Performs stretching centered on the top of the shoulders.	—	—	—	○
	Shoulder Stretch	Performs stretching centered on the shoulder blade area.	—	—	—	○

(*) Special operations cannot be combined with basic operations or rolling.

Leg Mechanism

	Function	Action	Up/Down Adjustment	Intensity Adjustment
Basic Operations	Soles	Performs a massage to the soles of the feet.	○	○
	Calves	Performs a massage to the calves.	○	○
	Thighs	Performs a massage to the thighs.	○	○
Rolling	Full Body Rolling	Performs stretching of the lower body using up and down movements.	○	○
	Partial Rolling	Performs partial stretching of the lower body using up and down movements.	○	○


(*) Area-specific operations and rolling cannot be combined.

Air

Function	Action	Intensity Adjustment
Arm Air	Performs an air massage on the right and left forearms alternately.	○

TROUBLESHOOTING

If there is still a problem after trying the solutions given in this troubleshooting guide, stop using the chair immediately to prevent accident or injury, and unplug the power plug from the wall outlet. Contact the vendor for assistance.

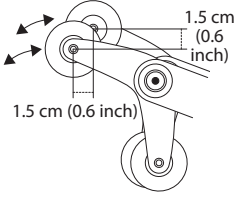Problem	Cause & Solution	This is part of the design and is not a malfunction.
<p>The left and right roller mechanisms are out of alignment.</p>	<p>The rollers are designed with an alternating tapping mechanism, so in the stopped position they are sometimes out of line. This is part of their design and there is nothing wrong with the chair.</p> 	
<p>The rollers moved to the head and leg areas, and then stopped.</p>	<p>The auto timer will cause the mechanism to move to the head and leg areas when the massage is finished, so as not to interfere with your movements when getting out of the chair.</p>	
<p>During kneading, sometimes rollers skip.</p>	<p>This is due to the rollers being caught in the boundaries between locations on the mechanism subjected to a large load and locations not subjected to a large load when a person is sitting in the chair. Individual differences such as body weight or body shape could also be responsible for jamming of the rollers.</p>	
<p>The POWER button of the remote control and the program button flash.</p>	<p>Set the main POWER switch located at the back of the chair to OFF, and then back ON after about 5 seconds.</p>	<p>If there is still a problem after the check, stop using the product immediately.</p> <p>TROUBLESHOOTING</p>
<p>The automatic shiatsu point locator function is not working properly. (The height of the rollers does not match the shoulders.)</p>	<p>Sit further back in the chair so that your head rests firmly on the headrest. You can adjust the shoulder position with the roller buttons immediately after the shiatsu points are located.</p>	
<p>Remote control buttons do not seem to be operating normally. Or the power does not turn on.</p>	<p>Check for the following:</p> <ol style="list-style-type: none"> ① Is the power plug fully inserted into the wall outlet? ② Is the main POWER switch located at the back of the chair set to ON? ③ Are you pressing two buttons at the same time? 	
<p>The power cord or plug is abnormally hot.</p>	<p>Stop using immediately.</p>	

Anyone other than an authorized serviceperson must not repair or disassemble.

Contact your local distributor.

TROUBLESHOOTING

Noise during chair operation

The following noises are made during operation.

Noises during tapping
Sound of the rollers rubbing against the seat cover during kneading
Sound of raising and lowering the rollers, sound of roller operations
Sound of the air massage cushions
Sound of the air pump
Sound of air discharge
Sound of raising and lowering the reclining seat back

*The sounds above do not affect performance or operation.

TROUBLESHOOTING

REGULAR MAINTENANCE FOR SAFETY

Regular Maintenance Will Preserve the Life of Your Chair

Have you experienced any of these problems?

- The chair does not start with the Main POWER switch set to ON.
- The power cord has been cracked or scratched.
- The power cord or plug becomes abnormally hot.
- The chair produces a burnt smell, abnormal sound or vibration.
- The auto timer function does not work in some cases.
- Other failures.

If you find anything listed on the left, stop using and unplug the power cord from the outlet to prevent a failure or accident. Be sure to contact your local distributor for an inspection.

To use safely

The fabric in the active area is a consumable part and its service life is 3 years. (Conditions of use: used 30 minutes a day, every day.)

For safe use, periodic inspection by your local distributor is recommended before the end of service life.

Durable years vary depending on conditions of use.

After about 5 years of use, a professional inspection is recommended even if there are no symptoms or problems.

For the inspection or repair fees, contact your local distributor.

REGULAR MAINTENANCE FOR SAFETY

FAQ

Q Can I put the chair onto a wooden floor or a heated floor?

A Yes.

However, because the chair could damage the floor surface, we recommend laying a carpet or something similar under the chair. (Same applies to other floor surfaces)

Q Can someone who is under 150 cm (5 feet) or over 180 cm (6 feet) use the chair?

A Yes.

The shoulder positions may not be correct during shiatsu point searching. If necessary, adjust the shoulder positions after shiatsu point searching.

Q Can I use the chair even if I am overweight?

A We recommend the chair be used by people weighing up to 100 kg (220 lb).

If you weigh more than that, try sitting slightly up in the chair, and not reclining too far.

Q I spilled water on the chair. What should I do?

A There is a danger of an electric shock. Immediately stop using the chair and perform the following.

- Set the main POWER switch located at the back of the chair to OFF.
- Unplug the power plug from the wall outlet. After performing the above, contact your local distributor for assistance.

Q How do I intensify the massage?

A Perform the following:

- Sit back further in the chair so that your weight is against the reclining seat back.
 - Set the mechanism intensity to High with the mechanism intensity button.
- With the methods above, you can intensify the massage.

Q How do I make the massage lighter?

A Perform the following:

- Attach the buffer pad.
 - Set the mechanism intensity to Low with the mechanism intensity button.
- With the methods above, you can make the massage more gentle.

Q Does the reclining seat back move automatically after a program has finished?

A The reclining seat back rises automatically after a preset program has finished.

However, when the automatic rising of the reclining seat back is canceled with the cancel button, the reclining seat back does not rise.

CARE AND STORAGE

Storage

- After wiping off any dust or dirt, store your chair in a place with low humidity.
- If the chair is left unused for a while, put a cover on it that will protect it from dust and dirt.
- Pets can sometimes chew on power cords, so keep pets away from the chair even when it is being stored.

⚠ Caution

- Do not install the chair where it can come into contact with direct heat, such as direct sunlight or a radiator.

Care and Maintenance

⚠ Caution

- Before cleaning the chair, make sure to unplug the power plug from the wall outlet. Do not plug or unplug the power plug with wet hands.

[Caring for the Pads and Covers]

Soak a piece of soft cloth in lukewarm water in which a small amount of mild detergent has been added. Then thoroughly wring it out before cleaning off any spots or grime.

⚠ Caution

- The fabric color may change if the chair is in contact with vinyl covers or wall coverings for a prolonged period of time.

[Caring for the Remote Control]

Wipe dirt off the remote control with a dry cloth.



- Never use a moistened or wet cloth. → May result in a mechanical failure.

[Caring for Other Plastic Parts]

Soak a soft cloth in lukewarm water mixed with a small amount of mild detergent, thoroughly wring it out, and wipe off any dirt. Then wipe with a dry cloth so that no detergent remains.

- Do not use alcohol-based or other household cleaning products or abrasives because they can scratch or crack the finish or cause color to fade.

CARE AND STORAGE

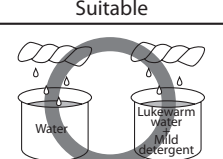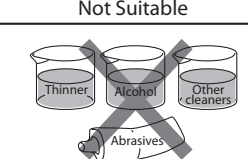Suitable	Not Suitable
 <p>Water</p> <p>Lukewarm water Mild detergent</p>	 <p>Thinner</p> <p>Alcohol</p> <p>Other cleaners</p> <p>Abrasives</p>

AFTER SALES SERVICE

1. Warranty

Please read the contents carefully and keep it in a safe place. Please complete and mail in the warranty card that was included with your chair.

If the chair is to be used outside the home (in a commercial setting, for example), the regular warranty is null and void. Please consult with your local distributor for policies regarding commercial use.

2. Ordering Repairs

Before ordering repairs or service, use the Troubleshooting guide on pages 22 - 23 to see if the problem can be solved easily. If the problem persists, contact your local distributor for assistance. Have the following information ready.

- 1) Your name, address and telephone number
 - 2) Vendor name, model number: See the specification field in this Operating Manual, or the company plate affixed to the chair.
 - 3) Serial No.: See the company plate affixed to the chair.
 - 4) Purchase date and proof of purchase/receipt
 - 5) Malfunction condition (give as much detail as possible)
- Example: When the remote control buttons flash as shown in the figure on the right, specify which program buttons are flashing.

- If the warranty is valid:
Please show your proof of purchase/receipt when you request a repair. In some case a charge will be applied even if the warranty period has not elapsed. Read the warranty certificate carefully.
- If the warranty has expired:
Contact your local distributor. If a repair will maintain the function of the product, we can perform the repair for a charge if you want.
- Minimum storage period for service parts:
We will stock service parts for this electric massage chair for a minimum of six years after terminating the production of the model. (Does not include fabric.) The service parts are those necessary for maintaining the function of the product.
- For more information on after sales service:
Contact your local distributor.

SPECIFICATIONS

Name & Model Number	HCP-WG1000A	HCP-WG1000D	HCP-WG1000E
Power Supply	AC 120 V~ 60 Hz	AC 220 - 240 V~ 50 - 60 Hz	AC 110 V~ 60 Hz
Power Consumption	160 W	160 W	160 W
Rated Time	30 minutes		
Kneading Speed	[Back] From minimum 6 times/minute to maximum 20 times/minute (approx.)		
	[Legs] From minimum 6 times/minute to maximum 20 times/minute (approx.)		
Tapping Speed	[Back] From minimum 180 times/minute to maximum 500 times/minute (approx.)		
Roller Up/Down Speed	[Back] From minimum 3.0 cm (1.2 inch) /second to maximum 3.6 cm (1.4 inch) /second (approx.)		
	[Legs] From minimum 3.0 cm (1.2 inch) /second to maximum 3.6 cm (1.4 inch) /second (approx.)		
Back Stroke Range	Approx. 140 cm (55.1 inch) [Back] Approx. 75 cm (29.5 inch) (in partial roller massage: automatic repeating in range of approx. 30 cm (11.8 inch)) [Legs] Approx. 65 cm (25.6 inch) (in partial roller massage: automatic repeating in range of approx. 35 cm (13.8 inch))		
Roller 3D Movement	[Back] Approx. 4.5 cm (1.8 inch)		
	[Legs] Approx. 5.7 cm (2.2 inch)		
Air Pressure	Approx. 37 kPa (pump discharge port area)		
Heater	Maximum 45°C (113 °F)		
Preset Programs	Healthcare programs: 5 types		
	Focused programs: 3 types		
Manual Programs	15 types (automatic shiatsu point search included in some programs)		
Program Timer	Approx. 15 minutes (manual programs)		
Overuse Prevention Timer	Approx. 30 minutes (continuous operation)		
Reclining Angle	Approx. 118° to 153° from floor level continuous motion (Electric powered reclining)		
Chair Dimensions	Approx. 86 × 140 (158) × 108 (76) cm (W/D/H) (Approx. 33.9 inch × 55.1 (62.2) inch × 42.5 (29.9) inch) The depth and height figures in () represent the dimensions when fully reclined.		
Weight	Approx. 80 kg (176 lb)		
Outer Covering	Synthetic leather, polyester		
Supplied Items	Power cord, remote control pocket, buffer pad		

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Manufactured by FAMILY INADA CO., LTD.